GB Instructions for Use

Introduction:

This blood pressure monitor conforms to the European requirements that are the basis of the Medical Products Law (CE), as well as to the European Standard „Europanorm“ EN 1060, Part 1 „Non-Invasive blood pressure monitoring devices – General requirements“ and part 2 „Additional requirements for mechanical blood pressure monitoring devices“.

Regular calibration testing is to be carried out in accordance with the regulations existing in individual countries.

Important:

Keep this unit from being knocked or dropped, and protect it from dirt and moisture. Care has to be taken not to puncture the cuff with any sharp instruments (Scissors, Needles).

Do not inflate to above 300 mmHg!

Measurement time should be ca. 2 minutes maximum.

Between 2 measurements allow an interval of at least 2 minutes.

Placing the Cuff:

Place cuff in such a manner that the lower cuff edge finishes ca. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

Taking the Blood Pressure:

Inflate the cuff to above the systolic pressure. The air release speed can be adjusted by rotating the valve screw.

The World Health Organisation (WHO) recommends a deflation speed of 2 – 3 mmHg per second.

After the measurement is completed, fully open the air release valve to allow for the fast deflation of the cuff.

Note on Cuff Sizes:

The German League for the Fight against Blood Pressure recommends the following cuff sizes:

Patient	Upper Arm Circumference in cm	Rubber Part of Cuff in cm Width x Length (*)
Infant	8 – 13	5 x 8
Child	14 – 21	8 x 13
Adult	22 – 32	12 – 13 x 24
	33 – 41	15 x 30
	41 +	18 x 36

* The given lengths are minimum measurements.

Please use only **boso** cuffs.

Technical Data:

Measurement range: 0 – 300 mmHg

Accuracy of Pressure indication: ± 3 mmHg

Cleaning of Unit and Cuff:

- To clean the unit only use a dry, soft cloth.
- To clean the cuff – remove the rubber bag. The exterior of the cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

Additional Notes – nova 2 Wall Model:

Mount the Universal holder using the screws provided with the set. Remove knurled nut from unit. Place unit from above into universal holder. Fasten with knurled nut. The cuff can be stored in its space provided on the holder.

Additional Notes – nova 2 Wall Model for Rail Mounting:

Fasten Universal holder to the rail with the accompanying screws. The unit can be then be fixed to the rail clip for the standard rail systems used in hospitals. This is done by turning the wing nut clockwise. The cuff can be stored in the space provided on the holder.

Additional Notes – nova 2 Stand Model

Set up stand following the accompanying instructions. Fasten the Universal holder with the inner hexagonal screw M8, also included, onto the stand. Remove knurled nut from the unit. Fit unit from above into the Universal holder and fasten with knurled nut. The cuff can be stored in its respective part on the holder.

Guarantee / Customer Service:

There is a 2-year guarantee from date of purchase. The conditions for guarantee are set out on the accompanying guarantee card.

Any guarantee is valid only if the item for return is accompanied by the relevant guarantee card duly completed, signed and stamped by the authorised dealer.

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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**Bosch + Sohn GmbH u. Co. KG