**Blutdruckmessgerät
Tensiomètre
Sfigmomanometro
Sphygmomanometer
Tensiómetro**

**clinicus I, clinicus II, manuell,
rapid, roid I, roid II, solid, varius**

- Ⓓ Gebrauchsanweisung
- Ⓕ Mode d'emploi
- Ⓘ Manuale di istruzioni
- ⒼB User Instructions
- Ⓔ Manual del Usuario

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GB User Instructions

Introduction:

This blood pressure monitor conforms to the European requirements that are the basis of the Medical Products Law (CE), as well as to the European Standard „Europanorm“ EN 1060, Part 1 „Non-Invasive blood pressure monitoring devices – General requirements“ and part 2 „Additional requirements for mechanical blood pressure monitoring devices“.

All Calibration testing –at least every two years–, has to be carried out either by the manufacturers or the local authorised dealer in accordance with the relevant distributor directives.

Important:

Keep this unit from being knocked or dropped, and protect it from dirt and moisture. Care has to be taken not to puncture the cuff with any sharp instruments (Scissors, Needles).

Do not inflate to above 300 mmHg!

Measurement time should be ca. 2 minutes maximum.

Between 2 measurements allow an interval of at least 2 minutes.

Please use only bosso cuffs.

Placing the Cuff:

Place cuff in such a manner that the lower cuff edge finishes ca. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

Taking the Blood Pressure:

Inflate the cuff to above the systolic pressure. The air release speed can be adjusted by rotating the valve screw.

The World Health Organisation (WHO) recommends a deflation speed of 2 – 3 mmHg per second.

After the measurement is completed, fully open the air release valve to allow for the fast deflation of the cuff.

Technical Data:

Measurement range: 0 – 300 mmHg

Accuracy of Pressure indication: ± 3 mmHg

Cleaning of Unit and Cuff:

- To clean the unit only use a dry, soft cloth.
- To clean the cuff – remove the rubber bag. The exterior of the cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

Guarantee / Customer Service:

For details on the conditions of guarantee, please refer to the guarantee card supplied.

Any guarantee is valid only if the item for return is accompanied by the relevant guarantee card duly completed, signed and stamped by the authorised dealer.

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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Bosch + Sohn GmbH u. Co. KG
BOSCH + SOHN GmbH u. Co. KG