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Dear Customer,

We are happy that you have decided on a product from our range. Our name stands for high-quality and thoroughly tested products from the areas of warmth, blood pressure, body temperature and weight.

Please read these instructions for use carefully and follow the directions.

Best regards,
Your bosso Team

1. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this heated underblanket for the purpose described in these instructions.
- This heated underblanket is not intended for use in hospitals.
- This underblanket is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.
- Do not use this heated underblanket for helpless persons, small children or persons insensitive to heat.
- The electric and magnetic fields emitted by this electrical heated underblanket may possibly interfere with the operation of pacemakers. However, they are well below the permissible limits: electric field strength: maximum of 5,000 V/m, magnetic field strength: maximum of 80 A/m, magnetic flux density: maximum of 0.1 millitesla. Please therefore ask your doctor and the manufacturer of your pacemaker before using this product.
- This heated underblanket should
 - only be connected to the voltage specified on the appliance,
 - should only be operated with the switch indicated on the device (label).
 - not be switched on if it is folded or compressed,
 - not be tucked into the bed or laid around the mattress,
 - never be wedged (especially with adjustable beds or divided mattresses),
 - never have any sharp creases,
 - never be used with animals,
 - never be used when damp.

- While this heated underblanket is switched on
 - never place any objects on it (e.g. suitcases or laundry basket),
 - never place any heat source on it, such as a hot water bottle, heating pad etc.
- When in use, the electronic components in the switch of the heated underblanket make the switch slightly warm. The switch should therefore never be covered or placed on the heated underblanket when it is in operation.
- Do not pull, twist or make any sharp kinks in the cables.
- Do not stick pins, needles or any sharp objects into this heated underblanket.
- This heated underblanket may not be used by children unless the switch has been preset by a parent or a supervisory person, or the child has been sufficiently instructed in how to use this heated underblanket safely.
- Children should be supervised to ensure that they do not use the device as a toy.
- This heated underblanket must be checked frequently for signs of wear or damage. If there are such signs or if the heated underblanket has been used incorrectly, it must be taken to either the manufacturer or the dealer before further use.
- Repairs may only be carried out by specially trained personnel or at an authorised repair workshop, as special tools are required. Incorrect repairs may cause considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

2. Proper Use

This heated underblanket is designed for warming beds. It is not intended for use in hospitals or for commercial applications. In particular, it may not be used for infants, small children, persons insensitive to heat, helpless persons or animals.

3. Operation

3.1 Safety

This heated underblanket is equipped with a SAFETY SYSTEM. This electronic sensor technology prevents the entire surface of the heated underblanket from overheating by automatically switching off in the event of a fault. If the safety system switches off the heated underblanket automatically as a result of a fault, the temperature level display on the switch is no longer illuminated even when the heated underblanket is switched on. Please note that, after a fault has occurred, the heated underblanket can no longer be used for safety reasons and must be sent to the service address indicated.

3.2 Putting into operation

Starting at the foot end, spread out the heated underblanket flat onto your mattress. Then put on your bedsheet in the usual manner so that the heated underblanket is between your mattress and the bedsheet. Make sure that the heated underblanket is spread out entirely flat and that it will not rumple up or crease when in use.

We strongly recommend switching on the heated underblanket approx. 30 minutes before going to bed and covering it with the quilt to prevent the warmth from escaping.

3.3 Selecting temperature

The quickest way to heat up the heated underblanket is to first select the highest setting. If it is to be used for several hours, we then recommend setting the lowest temperature.
To operate the underblanket first connect the switch to the radiator by plugging in the connector.

3.4 Temperature settings

0 = Off
1 = Minimum heat
2 = Maximum heat
3 = Maximum heat

3.5 Additional instructions for bosotherm 2200

With the bosotherm 2200 electric underblanket, automatic-off switching takes place after 3 hours. When the electric underblanket is heating up, the red signal lamp in the switch is lit. After being switched off by the timer, the electric underblanket is no longer heated up and the red signal lamp starts to flash. To heat up the electric underblanket again, first set the switch to "0" (Off) for approx. 5 seconds and then select the temperature level you require. If the electric underblanket is no longer used after being switched off by the timer, switch it off ("0") or remove the power plug from the power supply.

4. Cleaning and Care

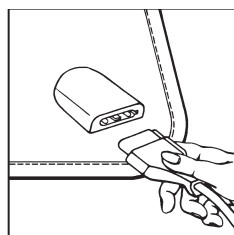
Before cleaning the heated underblanket, always unplug it from the mains and remove the plug-in connection together with the switch from the heated underblanket (see illustration).

Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary.

These electric underblankets are machine washable. Set the washing machine to a delicate washing cycle at 40 °C. Use a mild detergent with dosage according to the manufacturer's instructions. **Only reconnect the switch to the electric underblanket after the plug connection and the electric underblanket are completely dry.** Note

that the electric underblanket will suffer from too frequent washing. For its entire life, therefore, the electric underblanket should be washed not more than 5 times in a washing machine. Please note that the heated underblanket should not be dry-cleaned, wrung out, machine-dried, pressed or ironed.

Never switch on the heated underblanket to dry it.
Never use any clothes pegs or similar to secure the heated underblanket when drying.

5. Storage

If you are not going to use the heated underblanket for some time, we recommend storing it in its original packaging in a dry place without weighing it down.

Allow the underblanket to cool down beforehand.

6. Disposal

Please dispose of the heated underblanket in accordance with Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

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