(D) Gebrauchsanweisung

(GB) User Instructions

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(E) Manual del Usuario

boso
medilife PC3

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- 1 Blutdruckmessgerät
boso-medilife PC3
- 2 Batterien „Micro“ LR 03
- 1 Etui
- 1 CD profilmanager
- 1 PC-Verbindungskabel
- 1 Gebrauchsanweisung
- 1 Garantie-Urkunde

Garantie-Information

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Equipment and documentation supplied

- 1 Sphygmomanometer
boso-medilife PC3
- 2 Batteries 'micro' LR 03
- 1 Protective case
- 1 CD profile-manager
- 1 PC connecting cable
- 1 Instructions for use
- 1 Certificate of warranty

Guarantee information

For details on the conditions of guarantee, please refer to the guarantee card supplied.

A claim for guarantee can only be allowed if the Certificate of Warranty, completed and stamped by the dealer, is enclosed with the instrument.

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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| | | | |
|---|----|---|----|
| Important notes | 22 | The boson-medilife PC3 memory | 31 |
| General information | 23 | Installation of the boson profile-manager ... | 32 |
| Information about blood pressure | 23 | Connection to the PC | 32 |
| Blood Pressure (BP) values | 24 | Error messages | 34 |
| Preliminary remarks on measuring your own blood pressure | 25 | Cleaning and care of the unit | 34 |
| Preparing to measure blood pressure | 26 | Environmental safety notes | 34 |
| Inserting the batteries | 26 | Technical data | 35 |
| Setting the time and date on the boson-medilife PC3 | 27 | Re-Calibration Checks - Technical testing procedures | 36 |
| Preparing to measure blood pressure | 28 | | |
| How to wear the unit | 28 | | |
| Measuring your blood pressure | 29 | | |
| Reading the display | 30 | | |

Important notes

- All instructions of this manual are to be followed strictly. 
- Do not expose the unit to extreme temperatures and avoid exposure to prolonged direct sunlight. Guard against mechanical shocks (do not let the unit fall or be struck) and do not allow the unit to become soiled or wet.
- To prevent possible damage to the unit from leaking batteries, remove the batteries if the unit is not going to be used for a longer period.
- Medical Electrical Equipment needs special precautions regarding EMC. You will get appropriate information from the supplier on demand.
- The cuff of the sphygmomanometer is suitable for a wrist circumference between 13.5 and 21.5 cm.
- Take care to ensure that the cuff is not damaged by sharp or pointed objects (needles, scissors, etc.)

General information

This sphygmomanometer complies with the European directives underlying the guidelines for medical products (CE symbol), as well as the European Standard EN 1060 Part 1: 'Non-invasive Sphygmomanometers - General Requirements' and Part 3: 'Supplementary Requirements governing electronic blood pressure measurement systems'. Regular calibration testing is to be carried out in accordance with the regulations existing in individual countries.

The technical testing procedures are detailed on page 36.

Information about blood pressure

Blood pressure does not remain constant but is continually changing. In the morning the blood pressure is lower than in the evening and it is at its lowest during sleep. It is also influenced by eating, smoking, stress and bodily exertion.

High blood pressure is usually not immediately noticeable by the occurrence of particular complaints. If though, it goes unnoticed or untreated, it can lead to serious health problems. For this reason, regular blood pressure measurements as well as diagnosis and treatment by a doctor are so important.

A reliable assessment of blood pressure is only possible if you measure blood pressure regularly under comparable conditions.

However, measuring your own blood pressure is not a therapy. Never alter on your own accord any medication dosage that your doctor has prescribed.

Blood Pressure Values

To determine correct blood pressure, the following two values have to be ascertained:

- The **Systolic or upper blood pressure**. This is created when the heart muscle contracts and blood is being forced into the blood vessels.
- The **Diastolic or lower blood pressure**. This is created when the heart muscle extends and once again fills with blood.

Measurement values are expressed in **mmHg** (mm mercury column).

The World Health Organization (WHO) has set the following guideline for the assessing of blood pressure values:


| | Systolic | Diastolic |
|-------------------|-----------------|---------------|
| Too high | over 140 mmHg | over 90 mmHg |
| Normal–borderline | 130 to 139 mmHg | 85 to 89 mmHg |
| Normal | 120 to 129 mmHg | 80 to 84 mmHg |
| Optimum | up to 119 mmHg | up to 79 mmHg |

Whether medical treatment is necessary depends not only on the blood pressure but also on the risk profile of the individual patient. Please contact your GP if one of the values (SYS/DIA) constantly exceeds the limit (> 140 /> 90).

Preliminary remarks on measuring your own blood pressure

- Pregnant women, diabetics and persons suffering from liver disorders, irregular heartbeat or those with a cardiac pacemaker should consult their doctor before measuring their own blood pressure, as in such cases, variable and incorrect blood pressure readings may occur. The sphygmomanometer does not have any influence on the pacemaker.
- In cases of poor blood circulation or severe arteriosclerosis it is not appropriate to take measurements at the wrist. In these instances the blood pressure should be taken at the upper arm.
Even with persons not suffering from these conditions it can occur that there are differences between measurements at the wrist and the upper arm. Therefore it is recommended on occasions to take comparative measurements at the upper arm.
- The measurement should always be performed when relaxed and when sitting. Measurement should be carried out on the bare wrist.
- The arm must not be constricted as a result of clothing that has been pushed back or rolled up, and the arm should be kept still throughout the measurement.
- **When measuring, the wrist must be held at the same height as the heart!**
- It is recommended that the blood pressure is measured twice a day. In the morning after getting up and in the evening when the body is relaxed.
- Variations in blood pressure are perfectly normal. Even a repeat measurement can produce noticeable differences. One-off or irregular measurements are therefore of little value when trying to assess the actual blood pressure. An interval of at least two minutes should be observed, before repeating a measurement.

Preparing to measure blood pressure

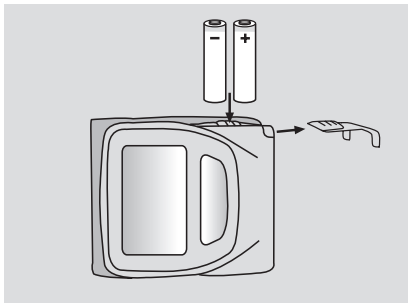Inserting the batteries

The battery compartment is located on the right upper side of the housing.

Insert 2 x 1.5 Volt micro time (LR03). Only use batteries of the given specifications and follow the polarisation markings in the battery compartment. Never mix old with new batteries, or batteries of different manufacture.

Slide the lid back into its original position.

- Remove batteries if the unit is not to be used for a longer period of time.
- The unit shows the following function symbols:

battery fully loaded.

battery partially loaded.

flashing! No further measurements possible; replace battery.

Used batteries and recharging units do not belong into the household bin. They can be deposited at special refuse centres for used batteries. Contact your townhall for information.


Important information on replacing batteries:

After replacement of batteries, please check date and clock settings.

Setting the time and date on the boso-medilife PC3

Your **boso-medilife PC3** unit is equipped with an internal time and date function. To set the time and date, press the SET button in the manner described below:

PC connection cable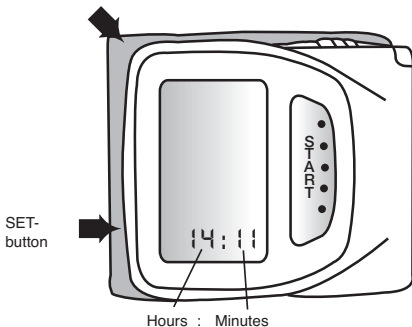

If the SET button is not pressed for approximately 30 seconds, the display returns automatically to its usual time display mode.

Time display mode

↓ press <SET> button once and release immediately
Date set mode (year flashes)...

→ ... set the year with the <START> button

↓ press <SET> button once and release immediately
Date set mode (month flashes)...

→ ... set the month with the <START> button

↓ press <SET> button once and release immediately
Date set mode (day flashes)...

→ ... set the day with the <START> button

↓ press <SET> button once and release immediately
Time set mode (hour flashes)...


→ ... set the hour with the <START> button

↓ press <SET> button once and release immediately
Time set mode (minute flashes)...


→ ... set the minute with the <START> button

↓ press <SET> button once and release immediately
Time display mode

Preparing to measure blood pressure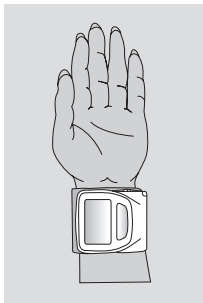

It is important to sit and relax for a few minutes before taking measurements which are as reliable as possible.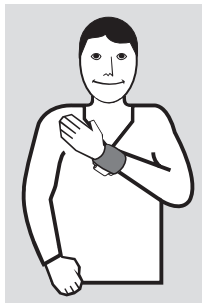

How to wear the unit


Wrap the cuff of the unit around your unclothed wrist. Please make sure, the display is positioned at your inner wrist (see picture).

Most persons have a higher blood pressure on their left wrist. Therefore the measurement should be taken on the left wrist. If the blood pressure is higher on the right wrist, the measurement should be taken on the right wrist.

Close the cuff with the velcro fastener. The cuff should fit snugly around the wrist. When relaxed, place your elbow on your chest with the wrist in the same height as the your heart.

Note on cuff size: The unit must only be used in combination with the CW51 cuff (wrist circumference 13.5 to 21.5 cm).

Measuring your blood pressure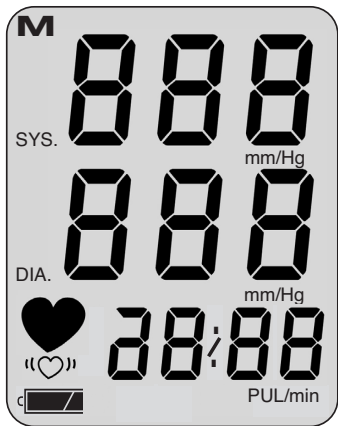


- Press the START button. All segments of the numerical display and all function symbols will be illuminated for a short period.
- The display indicates the pressure of the cuff as it rises.
- The unit inflates rapidly to a pressure of about 20 mmHg and then starts the measuring process.
- Now – at the latest - you should hold your hand absolutely still and stop talking.
- As soon as the unit registers the first beats, the ♡-symbol blinks in time with the pulse rhythm.
- After the measurement the built-in solenoid valve opens automatically to rapidly release the pressure in the cuff.

Reading the display

- The readings for systolic pressure, diastolic pressure and pulse are shown in the display. Measurements are automatically retained in the memory.
- Invalid measurements are not stored in the memory.
- If symbol "♥" appears following a completed measurement, it is recommended that the measurement is repeated while holding the arm motionless. A renewed display of symbol "♥" following the repeated measurement with the arm being held motionless can be an indication of irregular heart beat. Discuss this matter with your GP on your next visit.
- The unit switches off automatically after approximately one minute. To repeat a measurement, press the START button again.
- If no further measurement is intended, take off the cuff.

Aborting a measurement:


A measurement can be aborted at any time by pressing the START button. Then the cuff is automatically deflated (to start a new measurement, press the START button again).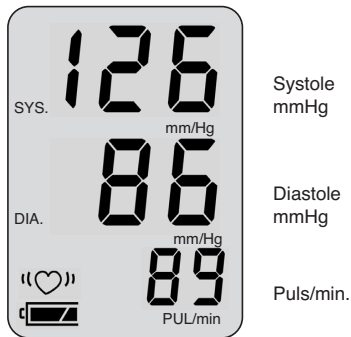

The bosomedilife PC3 memory

- The **bosomedilife PC3** stores up to 280 measurements. If the memory is full, (symbol **M** on the time display mode), the oldest measurement is cleared from memory and the most recent value stored.

You can recall stored readings by switching on the unit and pressing and holding down the START button until the last measurement that has been stored appears in the display (symbol **M**).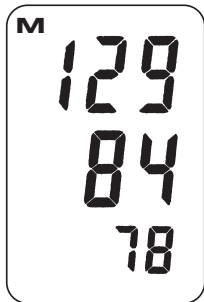

- Now all the stored measurement readings are shown successively for five seconds. (If no measurements are stored in memory, the systolic, diastolic and pulse readings all show the value '0' for three seconds.)
- By pressing the START button again, the memory display is interrupted and a new measurement started.
- The unit can be switched off by simply pressing the START-button.
- If all measurements of the memory have been displayed the unit switches off automatically after 5 seconds.

Erasing the memory

- Press the SET button for approximately five seconds until the **M** symbol starts to flash. The memory is now erased.

Installation of the bosoprofile-manager

- Put the enclosed CD into the CD drive of your computer
- The installation process gets automatically started by the autostart-function
- Choose the language of choice
- Follow the menu during the installation

A comprehensive instruction manual for the bosoprofile-manager is to be found on the enclosed CD.


Connection to the PC

- For a comprehensive assessment, the content of the memory is to be transferred via the **bosoprofile-manager (enclosed CD)** and the enclosed PC-connection cable to a computer.
This computer has to comply with the european standard EN 60601-1.
- Connect the bosomedilife PC3 with the PC connection cable to the USB port of your PC.

The data transfer mode gets activated by connecting the PC-connection cable into the port of the unit. 9 lines are to appear in the display.

- Follow the menu during the installation.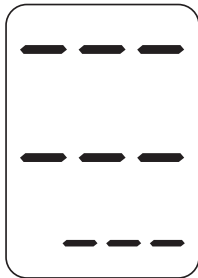

- Start the boso profile-manager via the icon “blood pressure measurement”
- Choose the part “self monitoring” via a mouse click on the left part of the monitor
- Connect the boso-medilife PC3 with the enclosed PC-connection cable with the USB port of your PC
- Through <Setup...><Interface> you choose the USB-connection and confirm with <OK>
- A new patient is created via the icon <new> in the register “Patient“ (mandatory are the patientnumber and the name)
- Choose the register “Edit“ and activate the icon <transfer measurements...>
- Press the start button on your boso-medilife PC3 within 6 seconds
- After successful data transfer you are asked to name the year when the last measurement took place. Please do this and confirm with <OK>
- Disconnect the PC-connection cable from the boso-medilife PC3.

- After successful data transfer delete the memory of the boso-medilife PC3 by pressing the SET-button until **M** flashes in the display

A comprehensive instruction manual for the boso profile-manager is to be found on the enclosed CD.

Attention:

No blood pressure measurement is possible during the data transfer mode.

Error messages

- If a problem should arise during a measurement which prevents a usable measurement from being made, an error number is displayed in place of the usual measurement result.
- Meaning of error messages:
 - Err 1/2 Cuff-pressure varies too much whilst measurement is taken → keep arm absolutely still - or -
No valid pulsations could be obtained.
→ Check positioning of cuff.
 - Err 3 Error during inflation.
→ Cuff is probably not tight enough.
 - Err 9 Communication error.
 - Err E ROM error.
 - Err
(field PUL/min) Pulse measurement error
→ Repeat measurement.
 - “(♥)” Irregular pulse or movement of arm during measurement
→ A renewed display of symbol “(♥)” following the repeated measurement with the arm being held motionless can be an indication of irregular heart beat. Discuss this matter with your GP on your next visit.

Cleaning and care of the unit

Please use only a soft, dry cloth to clean the unit.

Never use thinners, petroleum spirit or other aggressive cleaning agents.

Environmental safety notes

Packaging material and units that are no longer used should be recycled and not simply discarded.

Empty batteries and storage batteries should not be thrown away with household waste. They can be handed in at a battery collection point or disposed of as special waste.

Please contact your local authority for details.

Technical data

Measurement principle: oscillometric

Measurement range:

40 to 240 mmHg

40 to 200 pulses/min

Memory store: 280 measurements

Display: LCD

Operating conditions:

ambient temperature: 10°C – 40°C

relative humidity: 10 % – 85 %

Transport and storage conditions:

Temperature -5°C – 50°C

relative humidity: max. 85 %

Power supply:

DC 3V (2 x 1.5 V microcells alkali manganese, IEC LR 03)

Battery check:

indicator symbol in display

Weight: 82 g without batteries

Dimensions (W x D x H): 70mm x 55mm x 70mm

Classification: Type BF 

Clinical Test (DIN 58130):

Accuracy complies with EN 1060 part 3

Maximum deviation of cuff pressure: ± 3 mmHg

Maximum deviation of pulse rate: $\pm 5\%$

The right to alter technical specifications is reserved.

A) Functional testing

Functional testing of the unit can only be performed on human subjects or using a suitable simulator.

B) Testing of density of pressure circuit and divergence in the pressure indication

Please note:

- a) Prior to testing separate the cuff from the unit. To do so, remove the two connectors with a pointed instrument (i.e. tweezers). The two connectors can be found on the upper and lower side of the unit on each side of the cuff. After removal of the cuff, the air hoses become visible.
- b) To perform leak testing of the pressure circuit, T-connectors must be used to connect the rolled-up cuff and a pump bulb to the pressure circuit.

- c) If the pressure in measuring mode exceeds 320 mmHg, the quick-action air release valve triggers and opens the pressure circuit. If the pressure in testing mode exceeds 320 mmHg, the display blinks.

Testing

1. Connect lower hose of the unit with a pressure unit i.e. pump ball
2. Open battery compartment for more than 5 seconds. Hold START-button pressed down and insert batteries.
3. Release START-button.
4. The unit is now ready for testing. The displays of SYS and DIA now show the actual pressure.

5. Testing for divergence of pressure indication and density of pressure circuit (setting time for cuff at least 120 seconds) can now be carried out in the usual manner.
6. Switch off unit and reconnect the cuff to the unit.

C) Safety seal

To safeguard the correct calibration the two parts of the housing are joined together with a security seal.

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BOSCH + SOHN GmbH u. Co. KG