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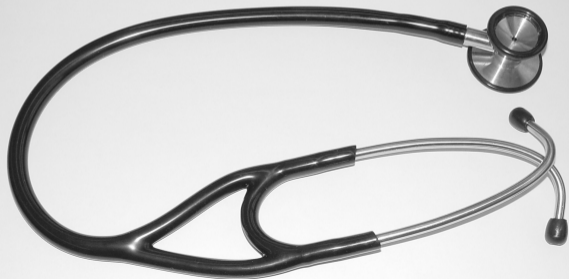
GB User Instructions

CE

boso

bososcope cardio

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stethoscope
spare eartips
spare diaphragms
instruction manual

General information

- Your boso-stethoscope conforms to the requirements of the European Directive 93/42/EWG (CE-mark).
- Please read this instruction manual carefully before using the stethoscope () and store it afterwards.
- Do not expose the stethoscope to extreme heat or cold.
- Do not steam sterilize the stethoscope.

- Do not put the stethoscope into cleaning liquid.
- The stethoscope can be damaged by sharp or pointed instruments.
- Do not stick the eartips too deep into the ear otherwise you may hurt yourself.
- Do not use the stethoscope without diaphragm.
- Only use spare parts of boso.

General use

- The stethoscope is a diagnostic instrument for auscultation.
- The stethoscope should be used by competent persons only.
- After placing the eartips into your ears, the chest piece has to be put on the relevant part of the body with careful pressure.

Using the chest piece

- The stethoscope can be adjusted from bell to the diaphragm or vice versa as the chest piece can be turned 180° both directions. A clicking sound can be heard as soon as the chest piece is engaged correctly on either the bell or the diaphragm.
- Depending on the body part which has to be auscultated either the diaphragm or the bell has to be used. The diaphragm is used for the auscultation of high frequencies, the bell is used for low frequencies.
- Place the eartips into your ears and gently sweep the diaphragm with the finger tip to test whether it is already active.

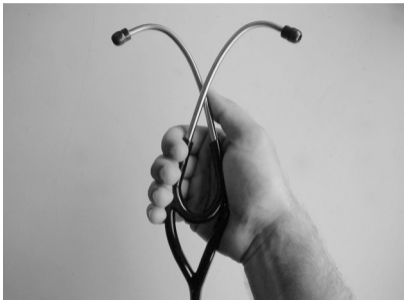
Adjustment of the binaural

- The binaural of your bososcope cardio is constructed and adjusted to fit comfortably to your ear and to conceal other noises from your ear.
- The ear tips should be placed into the ears in forward direction.
- If the tension of the binaural is not up to your personal need, it can be adjusted as follows:

too strong: hold the binaural as shown in the picture and pull the binaural slowly apart.

too soft: take the body of the binaural into your hand as shown in the picture and squeeze carefully so that both tubes cross over each other.

Repeat this until the perfect adjustment for your needs is achieved.

Changing the diaphragm

- To remove the diaphragm the chest piece has to be put onto a table with the diaphragm facing upwards. With your thumb and index finger you can roll the rubber ring aside until it slips over the edge of the chest piece.
- To fix the new diaphragm, put the chest piece onto a table with the diaphragm-side facing upwards. Then place the new diaphragm correctly. Now take the rubber ring and slip it back again. The easiest way to do this, is to take the rubber ring with its thinner brim of the ring facing downwards. Fix the rubber ring with your thumb and index finger in one position and slip the ring with the thumb of the other hand in a circular movement. Align the ring in such a way, that the diaphragm is held firmly.

Changing the eartips

- Soft eartips as well as hard eartips can be used on the binaural to adapt to the optimal acoustic sensitivity and comfort of the stethoscope. To change the eartips simply screw the required type onto the binaural.
- To avoid injuries, please check the correct position of the eartips on the binaural.

Cleaning

For cleaning use a soft cloth with mild soapy water or with alcohol (70% Isopropyl). The diaphragm must be cleaned only with a soft cloth with alcohol (70% Isopropyl).

Never use abrasive cleaning solution, thinner or benzene for cleaning.

Guarantee information

There is a 3 years factory warranty for defects in material and workmanship. From the warranty are excluded:

- all damages caused by improper handling
- accessories subject to normal wear
- damages already known to the buyer when the instrument is bought
- damages caused by the intervention of the buyer

A claim for guarantee can only be allowed within the 3 year period of warranty, if the date of purchase is proven (e.g. invoice).

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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Defects in material or workmanship will be serviced free of charge by us or a service partner authorized by ourselves. Any further claims are excluded.

BOSCH + SOHN GmbH u. Co. KG
Bahnhofstraße 64 · 72417 Jungingen · Germany
Telephone: +49 (74 77) 92 75-0 · Fax: +49 (74 77) 10 21
Internet: www.boso.de · e-Mail: zentrale@boso.de