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GB User Instructions

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Infrarotlampe

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Lampada a raggi
Infrarossi

Infra-red lamp

Lámpara de rayos
Infrarrojos

boso

bosotherm 4000

bosotherm 4100

Benutzung

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When using the device for the first time, check whether the light bulb is screwed tightly into place. Always ensure that the infra-red lamp is safely positioned on an even surface. During exposure to the rays, the distance to the lamp should be chosen in such a way that it feels pleasant.

Always close your eyes when exposing your face to the rays! The recommended duration of treatment is approx. 12 minutes.

Before cleaning and before replacing the light bulb, the device must be switched off, unplugged, and must have cooled down.

The infra-red lamp can be cleaned with a damp cloth. Do not use any aggressive cleaning or scouring agents. Ensure that no water gets into the inside of the device. Only use the device again when it is completely dry.

When replacing the light bulb, make sure that you use the same type of light bulb with the same power output. Your stockist can supply replacement light bulbs.

Important Information – store in a safe place for future reference

- Read this information carefully.
- Prior to using the infrared lamp, all packaging material must be removed.
- The infra-red lamp may only be connected to a mains supply that has the voltage indicated on the model identification label.
- **Close your eyes when exposing your face to the rays.**
- The infra-red lamp should not be used after you have taken painkillers or tranquilizers or consumed alcohol; this could reduce your sensitivity to heat. Consult your doctor before using the lamp.
- Children do not recognize the potential dangers involved in the use of electrical devices. In general, ensure that the device is not used by children without appropriate supervision.
- The infra-red lamp must not be either completely or partly covered when it is hot. Before touching the device, always disconnect it from the mains and allow it to cool.
- The maximum permissible proximity of the infra-red lamp to flammable objects is approx. 80 cm. Do not hang the device from the wall or from the ceiling.
- When the device is connected to the mains, it must not be touched with wet or damp hands; splashes of water must not make contact with the device. The infra-red lamp may only be used when it is absolutely dry.
- Protect the infra-red lamp against bumps and knocks.
- In the event of damage to the mains cable and the housing, consult your stockist; special tools are required in order to carry out repairs.

Guarantee and customer service information, Care of the Environment

For details on the conditions of guarantee, please refer to the guarantee card supplied.

A guarantee claim can only be accepted if the card, which has been filled-in and bears the stamp of the place of purchase, is sent in together with the unit.

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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Defective electronic devices do not belong into the household waste.

They can be deposited at special centres.

Please contact your local authorities for information.

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Bosch + Sohn GmbH u. Co. KG
BOSCH + SOHN GmbH u. Co. KG