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Dear Customer,

We are happy that you have decided on a product from our range. Our name stands for high-quality and thoroughly tested products from the areas of warmth, blood pressure, body temperature and weight.

Please read these instructions for use carefully and follow the directions.

Best regards,
The bosco Team

1. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this heating pad for the purpose described in these instructions.
- This heating pad is not intended for use in hospitals.
- This heating pad is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.
- Excessively long use can cause the skin to be burned.
- Caution! Never fall asleep while the heating pad is switched on.
- Do not apply the heating pad to parts of the body which are inflamed, injured or swollen. If in doubt, seek medical advice before use.
- The electric and magnetic fields emitted by this heating pad can interfere with the operation of pacemakers. However, they are far below the permissible limits: electric field strength: 5,000 V/m max., magnetic field strength: 80 A/m max., magnetic flux density: 0.1 millitesla max. Therefore, please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Heating pad
 - only connect to the voltage specified on the appliance,
 - never operate without supervision,
 - never switch on when folded or compressed,
 - never wedge between objects,
 - never make any sharp creases,
 - never use with animals,
 - never use when damp,
 - only use in conjunction with switch types specified on the heating pad.
- When in use, the electronic components in the switch of the heating pad make the switch slightly warm. The switch should therefore never be covered or placed on the heating pad when it is

being operated.

- Do not pull, twist or make any sharp kinks in the cables.
- Do not stick pins or pointed objects into this heating pad.
- Children should be supervised to ensure that they do not use the heating pad as a toy.
- This heating pad must be checked frequently for signs of wear or damage. If such signs are present, or if the cables are damaged, or if this heating pad has been used incorrectly, it must be taken to either the manufacturer or the dealer before further use.
- Repairs may only be carried out by specially trained personnel or at an authorised repair workshop, as special tools are required. Incorrect repairs may cause considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

2. Proper Use

This heating pad is designed for warming the human body. It allows you to apply targeted warmth. Heat promotes the circulation and relaxes the muscles. This heating pad is not intended for use in hospitals or for commercial applications. In particular, it may not be used on infants, small children, persons insensitive to heat, helpless persons or animals.

Our heating pads made of breathable microfleece are very adaptable and agreeable to the skin. They can also be used without their fabric cover. The detachable switch allows the pad to be cleaned in the washing machine (see cleaning instructions).

To avoid soiling or damaging the heating pad, we recommend using a fabric cover, particularly when applying essential oils and ointments.

When used for the first time, the heating pad may start to smell of plastic but this will soon go away.

3. Operation

3.1 Safety

This heating pad has a SAFETY SYSTEM. This electronic sensor technology prevents the entire surface of the heating pad from overheating by automatically switching off in the event of a fault. If the safety system switches off the heating pad automatically as a result of a fault, the indicator lamp is no longer illuminated even when the heating pad is switched on. Please note that, after a fault has occurred, the heating pad can no longer be used for safety reasons and must be sent to the service address indicated.

3.2 Selecting temperature

The fastest way to heat up the heating pad is to first set the switch to the highest setting. A lower setting can be selected later if required.

After switching on the heating pad, the setting starts flashing briefly, while the heating pad conducts a self-test. Then the setting remains illuminated continuously.

We recommend the lowest temperature setting for extended use of the heating pad.

3.3 Temperature settings

Setting: 0 Off

1 Minimum heat

2 Medium heat

3 Maximum heat

3.4 Automatic switch-off function

These heating pads have an automatic switchoff function which stops the heat supply approx. 90 minutes after switching on the pad. The illuminated function display then starts to flash. To operate the heating pad again, you must first set the switch to zero. After about 5 seconds, it can be switched on again.

If the heating pad is no longer used after the timed switch-off, it should be switched off (setting "0") or else unplugged from the mains.

3.5 Fast heating

These heating pads have a fast heating function which allows them to heat up quickly within the first 10 minutes.

3.6 Additional information bosotherm 1300

The particular shape of this heating pad was developed specially for use in the back and neck areas. In the neck region, the upper velcro fastener ensures that it stays in place. The rucksack fastener on the lower part of the heating pad enables you to adjust it to your body.

Press together the ends of the rucksack catch in order to close it. To open the catch, press both locking hooks simultaneously.

4. Cleaning and Care

Before cleaning the heating pad, always unplug it from the mains.

Open the zip in the cover and remove the heating pad. The textile covering can be cleaned in accordance with the symbols on the label.

Smaller stains on the heating pad can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary.

Do not use any cleaning agents containing solvents on the appliance.

Please note that the heating pad should not be dry-cleaned, wrung out, machine-dried, pressed or ironed.

Before washing, first remove the plug-in connection together with the switch from the heating pad (see illustration). Set the washing machine to a gentle washing cycle at 40°C.

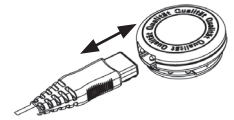
For ecological reasons, we recommend that the heating pad is only washed together with other textiles.

Use a washing powder or liquid for delicate fabrics and dose it according to the manufacturers instructions.

Do not reconnect the switch to the heating pad until the plug-in connection and the heating pad are completely dry. When replacing the heating pad in the cover, make sure that the cord comes out at the end of the zip, otherwise the zip cannot be closed.

Please note that excessive washing will wear out the heating pad. For this reason, the heating pad should only be machine-washed a maximum of 10 times throughout its entire life.

When drying the heating pad, do not use clothes pegs or anything similar, and never switch the heating pad on to dry it!

5. Storage

If you are not going to use the heating pad for some time, we recommend storing it in its original packaging in a dry place without weighing it down.

Allow the heating pad to cool down beforehand.

6. Disposal

Please dispose of the blanket in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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