# bosotherm basic

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Ⓜ User Instructions

Ⓝ Mode d'emploi

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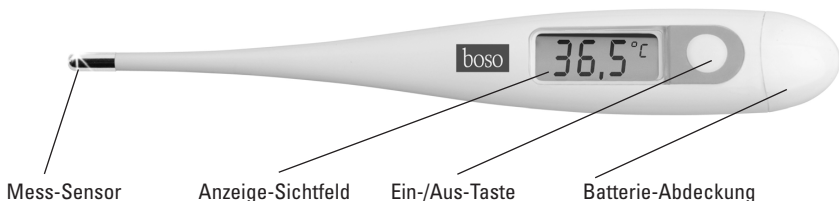
# bosotherm basic

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## Messarten

### Oral

60 ± 10 Sekunden  
Normal  
35,7 °C – 37,2 °C

### Rektal

60 ± 10 Sekunden  
Normal  
36,2 °C – 37,7 °C

### Axillar

100 ± 20 Sekunden  
Normal  
35,2 °C – 36,7 °C

## MESSARTEN

### • In der Mundhöhle (oral)

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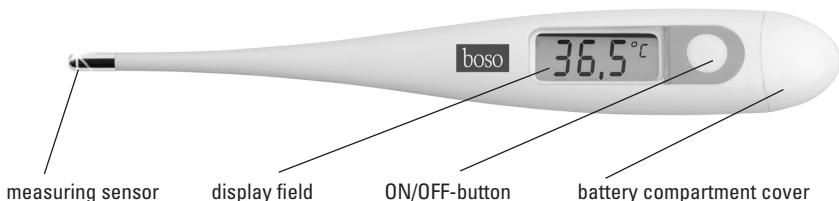
### MEASURING METHODS

#### • In the mouth (oral):

Put the thermometer underneath your tongue. The instrument must have good contact with the tongue. Close the mouth and breath calmly through your nose in order not to influence the measurement by the inhaled air. Minimum measuring time: 60 seconds

#### • In the anus (rectal):

This is the most reliable method and the most appropriate for babies and children. Carefully immerse the tip of the thermometer 2-3 cm into the anus. Minimum measuring time: 60 seconds

#### • Underneath the arm (axillary):

Under medical aspects this measuring method will lead to inaccurate results. Therefore it is not appropriate to get exact measurements. Minimum measuring time: 100 seconds

### IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is appropriate for the measurement of the body temperature only!
- Do not interrupt the measurement before the signalling tone [beep] indicates that the minimum measurement time is completed.
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from tremor and shock!
- Avoid ambient temperatures above 55°C. NEVER boil the thermometer!
- Only water or the disinfectants listed in the section «cleaning and disinfecting» are to be used for cleaning. The thermometer must be intact when it is immersed in liquid disinfectant.
- The thermometer has a long-life battery. For disposal, please comply with local regulations for special waste.
- Regular checks on the accuracy of the measurement system should be carried out in accordance with the valid directives of the individual countries, governing this issue.
- Medical Electrical Equipment needs special precautions regarding EMC. You will get appropriate information from the supplier on demand.

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CE 0124

## TURNING ON THE THERMOMETER

To turn on the thermometer, push the ON/OFF-button; a short signalling tone [beep] indicates «thermometer ON.» A so-called LCD-test automatically takes place to check the display. Then a «L» and a flashing «°C» appear on the upper right of the display, if the ambient temperature is less than 32°C. Now the thermometer is ready for use.

## USING THE THERMOMETER

During a measurement, the temperature is displayed continuously and the «°C» symbol flashes. If the signalling tone sounds (beep-beep-beep-beep 10 times) and the «°C» no longer flashes the measurement is completed (the measured increase in temperature is now less than 0,1 °C in 15 seconds).

To extend the lifetime of the battery, turn off the thermometer after usage by pressing the ON/OFF-button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

## MEMORY OF MEASURED VALUES

When turning the thermometer on, the temperature of the last measurement will be displayed. After 3 seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

## CLEANING AND DISINFECTING

For cleaning of the instrument please use isopropyl alcohol (70%). For disinfecting wipe the instrument completely with Antifect Liquid (please obey 5 min. work-in-time).

## CHANGE OF BATTERY

As soon as the symbol " □ " appears on the display, the battery is weary and a new battery has to be inserted. Open the cover of the battery compartment and insert the new battery with "+"symbol facing upwards.

## TECHNICAL SPECIFICATIONS

Maximum thermometer

Measuring range: 32.0°C to 42.0°C

Temp.< 32.0°C:display «L» for low (too low)

Temp.> 42.0°C:display «H» for high (too high)

Accuracy: ± 0.1°C between 35,5°C and 42°C at an ambient temperature of 5°C to 35°C

Liquid crystal display (LCD) with three digits. Smallest unit of display: 0.1°C

Signalling tone sounds when thermometer is ready for use and when measurement is completed.

Memory: automatic memory of the last measured value.

Storing temperature: -20°C to +55°C

Battery: 1.5/1.55 V

Norm: IEC 60601-1:1998 + A1:1991 + A2:1995, IEC 60601-1-2:2001

## WARRANTY

This unit is under warranty for 2 years from the date of purchase.

Please submit the receipt, invoice or delivery note when claiming for warranty.

For both, warranty and repair, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to

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## CARE FOR THE ENVIRONMENT

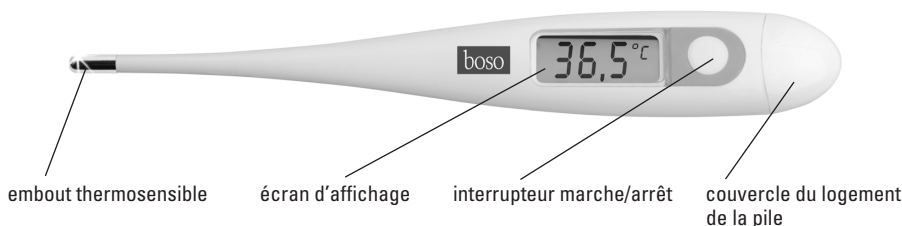
Used batteries and defective electronic devices do not belong into the household waste. 

They can be deposited at special centres. Please contact your local authorities for information.

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