DE Infrarotlampe Gebrauchsanweisung	2	TR Kızılötesi lamba Kullanma Talimatı	37
EN Infrared lamp Instruction for use.....	9	PL Lampa na podczerwień Instrukcja obsługi.....	44
FR Lampe infrarouge Mode d'emploi.....	16	NL Infraroodlamp Gebruiksaanwijzing.....	51
ES Lámpara infrarroja Instrucciones para el uso.....	23	PT Lâmpada de infravermelhos Instruções de utilização.....	58
IT Lampada a infrarossi Istruzioni per l'uso.....	30	EL Λάμπα υπεράυθρων Οδηγίες χρήσεως	65

Inhalt

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Content

1. Signs and symbols 9	7. Changing the infrared lamp 14
2. Notes 9	8. Cleaning and care of the unit 14
3. Unit description 11	9. Disposal 14
4. Initial use 11	10. Technical specifications 14
5. Operation 12	11. Warranty 15
6. Protective eyewear 12	

Included in delivery:

- Infrared lamp
- Protective eyewear
- These operating instructions

1. Signs and symbols

The following symbols appear in these instructions or on the device.

	Observe the instructions for use		Protect from moisture
	Protection class II unit	 Note	Note on important information
	Caution: hot surface		Manufacturer
 Warning	Warning instruction indicating a risk of injury or damage to health		Disposal in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
 Important	Safety note indicating possible damage to the unit/accessory	 0,8 m	Distance between infrared heat lamp and irradiated body part
	This product satisfies the requirements of the applicable European and national directives.	 PAP	Dispose of packaging in an environmentally friendly manner
	Serial number		

2. Notes

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Safety notes

Warning

- This infrared lamp is only intended for radiating the human body.
- Always use the protective eyewear supplied during usage.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Always ensure that the appliance is on a secure and even surface.
- The lamp housing and the infrared lamp become very hot during operation. There is a risk of burning if these are touched. Always let the lamp cool down first before touching it

- Inflammable objects must be kept at a distance of at least 1.0 m from the infrared lamp. Do not hang the appliance on the wall or from the ceiling.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms.
- This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.
- Do not use on babies and small children.
- Children should be supervised to ensure that they do not use the device as a toy.
- This device is not intended for use on changing units, cots or similar.
- Do not use it on persons sensitive to heat.

The sensitivity to heat can be reduced or increased in the following cases:

- diabetic patients
- persons with sleepiness, dementia or difficulties concentrating
- persons with illness-related skin changes or persons with scarred skin areas in the application area
- persons with allergies
- children and older persons
- after consuming medications or alcohol
- Do not operate the device if you are wearing an implant, particularly if it is located in the area of the body to be treated. If you have any doubts, consult your doctor.
- If you are pregnant, do not operate the device near your abdomen, in order to prevent this area from overheating. If you have any doubts, consult your doctor.
- In case of acutely inflammatory processes, radiation should only be carried out after consulting a doctor.
- Always limit the period of use and check the reaction of the skin.
- Medications, cosmetics or food can lead to a over-sensitive or allergic reaction of the skin under certain conditions. Radiation must be ended immediately in this case.
- An appropriate distance between the infrared lamp and the desired treatment area must be maintained during treatment so that no skin irritation, such as tingling, itching, burning or too high heat sensitivity occurs. Stop the treatment if this occurs. Alternatively, increase the radiation distance and/or reduce the radiation duration.
- This device is not intended for radiation from above.
- The minimum distances described in these instructions for use must be adhered to.
- If the appliance is operated continuously, caution and attention are advisable - never fall asleep during radiation!
- Prolonged use can cause burns to the skin.
- Do not touch the infrared lamp or unscrew it when it is connected to the mains.
- Children do not understand the risks associated with electrical appliances. Make sure the appliance is not used by children without appropriate supervision.
- The appliance may be used only under supervision.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover up or pack away the unit while it is warm.
- Always remove the mains plug and let the unit cool off before you touch it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Protect the unit from strong impacts.
- Do not pull the mains plug from the socket by the cord.
- In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. If the power cord of the device is damaged, it must be replaced by the manufacturer or an authorised customer service technician in order to avoid hazard.
- Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- There is a risk of strangulation for babies and children who become trapped in the mains cable.

General notes

Important

- Product is not suitable for lighting rooms in private homes.
- This unit is not intended for commercial or clinical use, but only for individual, private household use!
- If you have health concerns of any kind, contact your general practitioner!
- Before you use the unit for the first time, remove all packaging materials.
- Lighting is excluded from the warranty.

Repairs

⚠ Important

- Do not open the device. Please do not attempt to repair the unit yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.

3. Unit description

1. Light source
2. Lamp shade
3. Base
4. On/Off button + timer button
5. LED timer indication
6. First LED = readiness indicator/standby

4. Initial use

Setup

Remove the unit from the packaging. Remove all packaging material before using the appliance. Before use, check whether the infrared lamp is properly screwed in. Place the device on flat surfaces. The appliance has an adjustable lampshade. Set the angle of the fluorescent screen in accordance with your individual requirements.

Mains connection

Connect the unit only to the mains voltage listed on the type plate.

⚠ Note:

- Ensure that there is a socket near the place where the lamp will stand.
- Lay the mains cable in such a way that no one can trip over it.

Replacing the fuse (UK plug only)

If the fuse needs replacing, please observe the following:

- Remove the mains plug.
- Lift out the fuse holder from the inside of the mains plug using a flat screwdriver.
- Pull out the fuse from the fuse holder and replace it with a new fuse type: 3A (breaking capacity 6 kA at 240 V) in accordance with BS-1362 / SS167.
- Press the fuse holder containing the new fuse into the plug slot.
- Plug the mains plug into the socket and test to see if it works.

5. Operation

1	Switching on the lamp ▶ To connect to the mains, fully insert the plug into the socket. Once inserted, the „5 min“ LED will flash. This is the readiness indicator (standby). Press the On/Off button (4) once to switch on the lamp.								
2	Enjoying the light The recommended duration of treatment is maximum 15 minutes and should be limited to a maximum of two treatments a day. We recommend selecting a shorter radiation period to start with. However, the skin of some people may show an over-sensitive (e.g. with extreme redness, blistering, itching, heavy sweating) or allergic reaction to heat radiation, even when the lamp is used correctly.								
3	LED display/timer The infrared lamp indicates your current treatment time with the aid of 3 LEDs. The treatment time can be set at the following 3 levels. To scroll through the timer, hold down the On/Off button (4) until the required timer level lights up. The levels are as follows: <table border="1" data-bbox="163 475 631 603"><thead><tr><th>Treatment time</th><th>Number of lit LEDs</th></tr></thead><tbody><tr><td>5 minutes</td><td>1</td></tr><tr><td>10 minutes</td><td>2</td></tr><tr><td>15 minutes</td><td>3</td></tr></tbody></table> The lamp switches off automatically once the set treatment time has elapsed. To switch off the lamp manually, repeatedly press the On/Off button (4) until only the „5 min“ LED is flashing, the other LEDs are off and the lamp goes out.	Treatment time	Number of lit LEDs	5 minutes	1	10 minutes	2	15 minutes	3
Treatment time	Number of lit LEDs								
5 minutes	1								
10 minutes	2								
15 minutes	3								
4	Important instructions The distance between the infrared lamp and the radiated part of the body should not be less than the following: at least 32 in (80 cm). Always use the protective eyewear supplied during usage. The protective eyewear corresponds to the requirements of EN 171:2002 and serves to protect the eyes. CAUTION: Never look directly into the infrared light without protective eyewear. Basically, examine the radiated section of the body regularly. Discontinue radiation immediately if any signs of an over-sensitive or allergic reaction appear and consult a physician.								
5	Switching off the lamp CAUTION! The lamp is hot after being used. Let the lamp cool off first long enough before you put it away and/or pack it up! Please note that the mains cable must not be wound round the infrared lamp either for storage or when in use. Disconnect the mains plug from the socket to fully switch off the device.								

6. Protective eyewear

The product complies with the PPE regulation (EU) 2016/425 (reference: OJ L 81, 31/03/2016, p. 51-98), the harmonised standard EN 166:2001 (reference: OJ C 209, 15/06/2018, p. 18) and the standard EN 171:2002.

Warnings and safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the product to third parties.

WARNING

- Keep packaging material away from children. There is the risk of suffocation.
- Children must not play with the protective eyewear.
- Before using the protective eyewear, check it for signs of external damage (e.g. cracks or holes). If the protective eyewear is damaged, you must NOT continue using it.

- When wearing the protective eyewear, people with sensitive skin may have an allergic reaction to the materials of the eyewear that may come into contact with the skin.
- Make sure that the protective eyewear does not come into contact with grease, oils or alcohol (e.g. cleaning products or facial toners).
- Please note that the protective eyewear is opaque, which means it will restrict your spacial awareness.
- Do NOT operate the infrared lamp whilst wearing the protective eyewear.
- Do NOT move about whilst wearing the protective eyewear.
- Make sure that you do NOT reduce the distance between you and the infrared lamp whilst wearing the protective eyewear.
- Use the protective eyewear only in quiet environments, so that you are not disturbed during use.

Intended use

The protective eyewear is exclusively intended for protecting your eyes against infrared radiation while using an infrared lamp. The protective eyewear provides protection against infrared radiation only. The protective eyewear does NOT provide protection against other types of radiation. Use the protective eyewear only with boson infrared lamps. Never use the protective eyewear with infrared lamps from other manufacturers.

Always use the protective eyewear directly over your eyes, not over another pair of glasses. The protective eyewear is intended for domestic/private use only, not for commercial use. It is free from pollutants as per the legal requirements. The material of the protective eyewear may initially smell. This smell will dissipate after a few uses.

Usage

1. Position the infrared lamp.
 2. Prepare the desired treatment distance.
 3. Switch on the infrared lamp.
 4. Sit down in preparation for the application.
 5. Hold the protective eyewear to your eyes and pull the rubber band over your head. To make the rubber band longer/shorter, pull on the strap at the side.
 6. To end the application, first take off the protective eyewear. Then switch off the infrared lamp.
- Replace the product after a period of use of 3 years.

Storage

If you do not plan to use the protective eyewear for an extended period of time, we recommend storing it in its original packaging in a dry environment, ensuring that it is not weighed down by other items.

Cleaning

Clean the protective eyewear from time to time. Use a slightly damp, soft cloth to clean it. Do not use any sharp or abrasive cleaning products.

Technical data

Model	SEP 01
Protection level	4-10 in accordance with EN 171:2002
Protective eyewear designation	4-10 HAD CE The code for the lenses is comprised of the following: - Initial number for IR protection filter (1st part of code) - Protection level number (2nd part of code) - Manufacturer's code (3rd part of code) - CE mark (4th part of code)
Light transmission level	Max. 0.023% Min. 0.0085%
Maximum average spectral transmittance value with infrared	780 – 1400 nm: 0.050% 780 – 2000 nm: 1.3%

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Notified body:
ECS European Certification Service GmbH
Hüttfeldstraße 50
73430 Aalen, Germany
Identification number: 1883

EU declaration of conformity: <http://sanitas-online.de/web/de/landingpages/cedeclarationofconformity.php>

7. Changing the infrared lamp

The device must be switched off and allowed to cool and the mains plug pulled out before cleaning and changing the infrared heat lamp.

Please note that only infrared lamps of the original type may be used as replacements.

Replacement infrared lamp (type):	Specifications:	Item number:
Infrared PAR 38 (150 W)	230V ~ 50 Hz	616.51

Warning

- Do not touch the infrared lamp or unscrew it when it is connected to the mains.
- Unplug the device, to avoid the risk of electrocution.
- In the interest of environmental protection, do not dispose of the device in household waste.

8. Cleaning and care of the unit

The unit should be cleaned occasionally.

Important

To avoid odours caused by fluff and dust, we recommend regularly cleaning the device as follows:

- The device must be switched off and allowed to cool and the mains plug pulled out before cleaning and changing the infrared heat lamp.
- Ensure that no water penetrates inside the unit!
- Do not wash the unit in a washing machine!
Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not use abrasive detergents and never immerse the unit in water. Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit.
- The unit must be operated only when it is completely dry.

9. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

10. Technical specifications

Mains connection	220-230 V~ / 50-60 Hz
Power consumption	150 W
Max. radiation intensity	550 W/m ² at distance of 80 cm
Dimensions (W x H x D)	145 x 220 x 175 mm
Weight	approx. 800 g
Operating conditions	Temperature: +5°C to +35°C; Relative humidity: 15%-93%
Storage and transport	Temperature: -25°C to +70°C; Relative humidity: ≤93%
Expected operating life of the device	7 years
Standby consumption	< 0.5 watts

Subject to technical modifications. The serial number is located on the device or in the battery compartment.

11. Warranty

We offer a 3-year product warranty against material and production faults. The warranty does not cover:

- Damage due to improper operation
- Wearing parts
- Deficiencies already known to the customer upon purchase
- Cases of personal negligence on the part of the customer
- Outside interference

The warranty does not affect the customer's statutory rights. To assert a warranty claim within the warranty period, the customer must provide proof of purchase.

The warranty claim must be submitted within a period of 3 years from the date of purchase to bosco GmbH+Co. KG, Bahnhofstr. 64, D-72417 Jungingen (Germany). In the event of a warranty claim, the customer reserves the right to have the goods repaired at our own workshop or at a workshop authorised by us. No further rights are granted to the customer (on the basis of the warranty).

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