boso bososcope cardio

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Ⓓ

2

User
Instructions

ⒼⒷ

12

Mode
d'emploi

Ⓕ

22

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Ⓘ

32

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Ⓔ

42

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D

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GB

General information

- Your boso-stethoscope conforms to the requirements of the European Directive 93/42/EWG (CE-mark).
-  Please read this instruction manual carefully before using the stethoscope and store it afterwards.
- Do not expose the stethoscope to extreme heat or cold.
- Do not steam sterilize the stethoscope.

- Do not put the stethoscope into cleaning liquid.
- The stethoscope can be damaged by sharp or pointed instruments.
- Do not stick the eartips too deep into the ear otherwise you may hurt yourself.
- Do not use the stethoscope without diaphragm.
- Only use spare parts of bosco.

14 General use

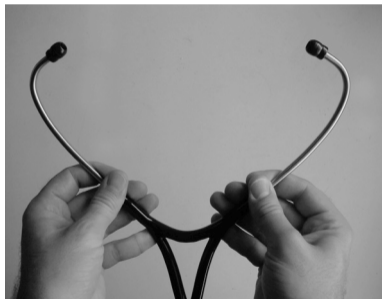
- The stethoscope is a diagnostic instrument for auscultation.
- The stethoscope should be used by competent persons only.
- After placing the eartips into your ears, the chest piece has do be put on the relevant part of the body with careful pressure.

- The stethoscope can be adjusted from bell to the diaphragm or vice versa as the chest piece can be turned 180° both directions. A clicking sound can be heard as soon as the chest piece is engaged correctly on either the bell or the diaphragm.
- Depending on the body part which has to be auscultated either the diaphragm or the bell has to be used. The diaphragm is used for the auscultation of high frequencies, the bell is used for low frequencies.
- Place the eartips into your ears and gently sweep the diaphragm with the finger tip to test whether it is already active.

16 Adjustment of the binaural

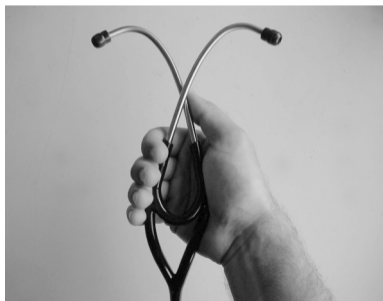
- The binaural of your bososcope cardio is constructed and adjusted to fit comfortably to your ear and to conceal other noises from your ear.
- The ear tips should be placed into the ears in forward direction.
- If the tension of the binaural is not up to your personal need, it can be adjusted as follows:

too strong: hold the binaural as shown in the picture and pull the binaural slowly apart.

Adjustment of the binaural

too soft: take the body of the binaural into your hand as shown in the picture and squeeze carefully so that both tubes cross over each other.

Repeat this until the perfect adjustment for your needs is achieved.

18 Changing the diaphragm

- To remove the diaphragm the chest piece has to be put onto a table with the diaphragm facing upwards. With your thumb and index finger you can roll the rubber ring aside until it slips over the edge of the chest piece.
- To fix the new diaphragm, put the chest piece onto a table with the diaphragm-side facing upwards.

Then place the new diaphragm correctly. Now take the rubber ring and slip it back again. The easiest way to do this, is to take the rubber ring with its thinner brim of the ring facing downwards. Fix the rubber ring with your thumb and index finger in one position and slip the ring with the thumb of the other hand in a circular movement. Align the ring in such a way, that the diaphragm is held firmly.

- Soft eartips as well as hard eartips can be used on the binaural to adapt to the optimal acoustic sensitivity and comfort of the stethoscope. To change the eartips simply screw the required type onto the binaural.
- To avoid injuries, please check the correct position of the eartips on the binaural.

Cleaning

For cleaning use a soft cloth with mild soapy water or with alcohol (70% Isopropyl). The diaphragm must be cleaned only with a soft cloth with alcohol (70% Isopropyl).

Never use abrasive cleaning solution, thinner or benzine for cleaning.

Disinfection

For disinfectant wipes (at least 5 minutes exposure time) of the device, we recommend the disinfectant liquid Microzid Liquid Sensitiv (Schülke & Mayr).

We give 3 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear, transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unautho-

rized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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BOSCH + SOHN GmbH u. Co. KG