# bosotherm flex

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ⒺN User Instructions

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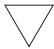
**BOSCH  
+SOHN**

boso

**(DE) Symbole auf dem Thermometer / (EN) Symbols on the thermometer / (FR) Symboles sur le thermomètre / (ES) Símbolos en el termómetro / (IT) Simboli sullo termometro**

Symbol / Symbol / Symbole / Símbolo / Simbolo	Funktion – Bedeutung / Function/meaning Fonction/signification / Función/significado Funzione/Significato
	Ein-/Aus-Taste ON/OFF-button Interrupteur marche/arrêt Botón ON/OFF Copercio
	Keine weitere Messung möglich. Batterien ersetzen. No further reading possible. Replace battery. Une mesure n'est plus possible. Remplacer la batterie. No son posibles más mediciones. Sustituir las baterías. Impossibile continuare la misurazione. Sostituire le batterie.
	Gerät vom Typ BF Type BF equipment Appareil de type BF Aparato del tipo BF Apparecchio di tipo BF
	Lotnummer Lot Number Numéro de Lot Número de Lot Numero di Lot
	Hersteller Manufacturer Fabricant Fabricante Produttore
	Gebrauchsanweisung lesen Read instructions for use Lire le mode d'emploi Leer las instrucciones de uso Leggere le istruzioni per l'uso
	Gerät darf nicht über den Hausmüll entsorgt werden. Device must not be discarded with household waste. L'appareil ne doit pas être jeté à la poubelle. El aparato no se debe eliminar en la basura doméstica. L'apparecchio non deve essere smaltito con i rifiuti urbani generici.
--- °C	Temperatur unter 32 °C Temperature under 32 °C Température inférieure à 32 °C Temperatura inferior a 32 °C Temperatura inferiore a 32 °C
H °C	Temperatur über 42 °C Temperature over 42 °C Température supérieure à 42 °C Temperatura superior a 42 °C Temperatura superiore a 42 °C
ERR	Interner Gerätefehler Internal error Erreur interne Error interno Errore interno

CE 0124

Gerät ist konform mit der europäischen Medizinprodukterichtlinie.  
Device complies with the European Medical Devices Directive.  
L'appareil est conforme à la directive européenne « Dispositifs médicaux ».  
El aparato cumple la Directiva sobre Productos Sanitarios europea.  
L'apparecchio è conforme alla direttiva europea sui dispositivi medici.

# bosotherm flex

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Rektal

Oral/Axillar

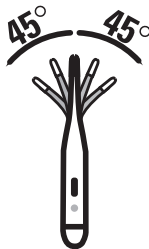
Signalton

100% wasserdicht (IP 67)

Speicher

## bosotherm flex

Mess-Sensor

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Ein-/Aus-Taste

Batterie-Abdeckung

### Messarten

#### Oral

30 Sekunden  
Normal  
35,7 °C – 37,2 °C

#### Rektal

10 Sekunden  
Normal  
36,2 °C – 37,7 °C

#### Axillar

30 Sekunden  
Normal  
35,2 °C – 36,7 °C

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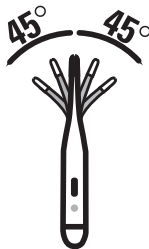
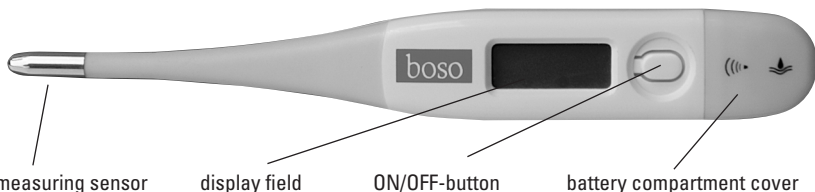
#### Axillary

30 seconds  
normal temp.:  
35,2°C – 36,7°C

Before each use of the thermometer, make sure that it is not damaged.

### MEASURING METHODS

#### • In the mouth (oral):

Put the thermometer underneath your tongue. The instrument must have good contact with the tongue. Close the mouth and breath calmly through your nose in order not to influence the measurement by the inhaled air. Minimum measuring time: 30 seconds

#### • In the anus (rectal):

This is the most reliable method and the most appropriate for babies and children. Carefully immerse the tip of the thermometer 2-3 cm into the anus. Minimum measuring time: 10 seconds

#### • Underneath the arm (axillary):

Under medical aspects this measuring method will lead to inaccurate results. Therefore it is not appropriate to get exact measurements. Minimum measuring time: 30 seconds

### IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is appropriate for the measurement of the body temperature only!
- This device can be used at home and in hospital.
- The user should be at least 11 years old.
- The patient can be the user.
- Do not interrupt the measurement before the signalling tone [beep] indicates that the minimum measurement time is completed.
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from tremor and shock!
- Avoid bending of the tip more than 45°!
- Avoid ambient temperatures above 60°C. NEVER boil the thermometer!
- Only water or the disinfectants listed in the section «cleaning and disinfecting» are to be used for cleaning. The thermometer must be intact when it is immersed in liquid disinfectant.
- The thermometer has a long-life battery. For disposal, please comply with local regulations for special waste.
- Regular checks on the accuracy of the measurement system should be carried out in accordance with the valid directives of the individual countries, governing this issue.
- Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this thermometer. Therefore, a minimum distance of 11 feet should be kept from such devices.

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CE 0124

## PURPOSE

Determination of body temperature of persons

## TURNING ON THE THERMOMETER

The device must be installed and put into operation in accordance with the instructions in this manual.

To get help concerning usage or maintenance please contact your dealer or the manufacturer.

An unexpected operating condition or an incidence which has worsened the health condition or could have worsened the health condition should be reported to the manufacturer immediately.

To turn on the thermometer, push the ON/OFF-button next to the display; a short signalling tone [beep] indicates «thermometer ON.» A so-called LCD-test automatically takes place to check the display. Then «---» and a flashing «°C» appear on the upper right of the display, if the ambient temperature is less than 32°C. Now the thermometer is ready for use.

## USING THE THERMOMETER

During a measurement, the temperature is displayed continuously and the «°C» symbol flashes. If the signalling tone sounds (beep-beep-beep-beep 10 times) and the «°C» no longer flashes the measurement is completed (the measured increase in temperature is now less than 0,1 °C in 8 seconds).

To extend the lifetime of the battery, turn off the thermometer after usage by pressing the ON/OFF-button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

## REVIEWING THE LAST MEASUREMENT

Press ON/OFF button for 2 seconds, when turning on the thermometer. The last measurement will be displayed together with a «M» for memory.

After 2 seconds, the thermometer will automatically switch off.

## CLEANING AND DISINFECTING

For cleaning of the instrument please use isopropyl alcohol (70%). For disinfecting wipe the instrument completely with Antifect Liquid (please obey 5 min. work-in-time).

## CHANGE OF BATTERY

As soon as the symbol «▽» appears on the display, the battery is weary and a new battery has to be inserted. Open the cover of the battery compartment and insert the new battery with «+»symbol facing upwards.

## TECHNICAL SPECIFICATIONS

Maximum thermometer

Measuring range: 32.0°C to 42.9°C

Temp.< 32.0°C:display «---» for low (too low)

Temp.> 42.9°C:display «H» for high (too high)

Accuracy: ± 0.1°C between 34°C and 42°C at an ambient temperature of 18°C to 28°C

Operating conditions: environmental temperature +10°C to +40°C, relative humidity 15 to 95%

Self-test: automatic internal check at a test value of 37.0°C; if there is a deviation of >0.1°C, «ERR» (error) is displayed.

Liquid crystal display (LCD) with three digits. Smallest unit of display: 0.1°C

Signalling tone sounds when thermometer is ready for use and when measurement is completed.

Memory: automatic memory of the last measured value.

Storage conditions: environmental temperature - 25°C to +60°C, relative humidity 15 to 95%

Battery: 1,5 V, IEC-type LR/SR 41

1,000 measurement cycles (depending on frequency of use).

 Type BF applied part

Norm: EN12470-3, clinical thermometers

Protection against solid objects and water: IP67 (in accordance with IEC 60529)

Expected operational lifetime of the equipment: 10 years

## WARRANTY

This unit is under warranty for 2 years from the date of purchase.

Please submit the receipt, invoice or delivery note when claiming for warranty.

For both, warranty and repair, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to

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Warranty and repair work must be undertaken by trained and authorised personnel. Do not modify this equipment without authorization of the manufacturer.

## CARE FOR THE ENVIRONMENT

Used batteries and defective electronic devices do not belong into the household waste. 

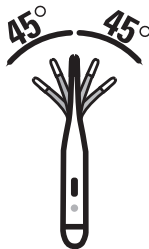
They can be deposited at special centres. Please contact your local authorities for information.

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