# boso

## **classic, classico**

- ⒹE Gebrauchsanweisung
- ⒺN Instructions for Use
- ⒻR Mode d'emploi
- ⒾT Istruzioni per l'uso
- ⒺS Instrucciones de Uso

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# **EN** Instructions for Use

## **Contents of the kit:**

- 1 blood pressure instrument with cuff
- 1 set of instructions for use
- 1 case

## **Symbols on the blood pressure monitor:**

<b>Symbol</b>	<b>Function/meaning</b>
<b>CE 0124</b>	Device complies with the European Medical Devices Directive
	Year of manufacture

## **Preliminary remarks:**

This blood pressure instrument complies with the international standard ISO 81060-1.

All Calibration testing –at least every two years–, has to be carried out either by the manufacturers or the local authorised dealer in accordance with the relevant distributor directives.

A modification of the device and/or the accessories is not permitted. This can lead to measurement errors.

The device can be used by any user skilled in auscultatory blood pressure measurement.

## **Purpose:**

Non-invasive recording of systolic and diastolic blood pressure in humans. The instrument can be used with any arm circumference as shown on the corresponding cuff.

## **Important:**

Protect your blood pressure monitor from damp and dirt. Care has to be taken not to knock or drop the unit and that the cuff does not get damaged by sharp objects (needles, scissors etc.).

Do not inflate to above 300 mmHg.

Individual measurements should not exceed a maximum of 2 minutes.

Between measurements there should be an interval of at least 2 minutes.

The patient must follow these basic rules when having his/her blood pressure measured:

- sit comfortably
- not cross his/her legs
- rest his/her back and arms on the chair or another surface
- have the centre of the cuff placed on the upper arm at the level of the heart
- relax as much as possible and keep silent while his/her blood pressure is being measured
- be calm for 5 minutes before the first reading

The following phases are recommended in the case of auscultatory measurement

- adults: phase V (K5)
  - children aged 3 to 12: phase IV (K4)
  - pregnant women: phase V (K5), except when sounds are audible when the cuff is deflated: in this case phase IV (K4)
- Korotkoff sounds should be used.

K5 is the point at which tones heard through the stethoscope are no longer audible

K4 is the point at which tones heard through the stethoscope change from a clear sound to a muffled sound.

### **Inscriptions:**

“mmHg” is the unit in which blood pressure is measured

“shock protected” = the instrument has been tested and found to work correctly after being dropped from a height of up to one metre.

### **Fitting the Cuff:**

The cuff must not be fitted over wounds as this could lead to further injury.

Blood pressure must always be measured on the arm with the higher pressure values.

To do this, first measure blood pressure on both arms and subsequently always on the arm which produced the higher reading.

Place cuff in such a manner that the lower cuff edge finishes ca. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

### **Taking the measurement:**

Fully close the valve by turning the air release screw to the right. Inflate the cuff to above the systolic pressure. By turning the air release valve the air release speed can be adjusted. The World Health Organisation (WHO) recommends a deflation rate of 2 – 3 mmHg per second. To avoid a reading error, read the pressure values perpendicular to the scale.

After the measurement is completed open the release valve to fully deflate the cuff.

### **Cuffs:**

**Only use the prescribed boso cuffs with their special connector.**

The cuff must be selected such that it fits the arm circumference.

### **Technical Data:**

Measuring range: 0 – 300 mmHg

Accuracy of indication of pressure:  $\pm 3$  mmHg

Storing conditions:

$-30^{\circ}\text{C} - 70^{\circ}\text{C}$ , 15 – 85% rel. humidity

Operating conditions:

$10^{\circ}\text{C} - 40^{\circ}\text{C}$ , 15 – 85% rel. Humidity

### **Cleaning of unit and the cuff:**

- To clean your unit it is only necessary to use a soft dry cloth.
- To clean the cuff, first remove the rubber bag. The outer cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

### **Disinfection:**

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend the disinfectant liquid Microzid Liquid Sensitiv (Schülke & Mayr).

To disinfect the cuff, we recommend spray disinfection.

## **Warranty Conditions:**

We give 2 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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## **Disposal:**

Please ensure that the device is disposed of according to all regional and national environmental regulations.

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