boso medicus system

**BOSCH
+SOHN**

boso

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(DE) 2

**User
Instructions**

(EN) 50

**Mode
d'emploi**

(FR) 98

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(IT) 146

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(ES) 194

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Table of contents

Product features	52
Contents of package	53
Symbols on the blood pressure monitor and in the app	54
Symbols on the power supply unit.....	57
Quick guide.....	58
Introduction.....	59
Purpose	60
Blood pressure values	61
Systolic and diastolic blood pressure value	61
WHO blood pressure cut-off values.....	61
Starting up	63
Inserting batteries	63
Battery status.....	64
General instructions for self-measurement	65
General Notices for Bluetooth Connection.....	67
Preparing to measure your blood pressure	69
Note on cuff sizes.....	69

Table of contents

Attaching the cuff	70
Measuring your blood pressure	73
Interrupting a measurement	74
Measurement display	75
Set up app and create connection	77
Measuring and transferring blood pressure	81
Evaluating and sending measurements	83
Deleting measurement values	85
Mains operation	86
Error messages	87
Cleaning and Disinfection	90
Warranty Conditions	91
Customer service / Disposal	92
Technical data	93
Calibration checks - Testing instructions (for trained specialist staff only)	96

Product features

Contents of package

1 **boso medicus system**
blood pressure monitor

4 LR 6 (AA) batteries

1 CA 04 universal cuff

1 User
instructions

1 Bag

Symbols on the blood pressure monitor

Symbol	Function/meaning
	START/STOP button
	Direct current
	Serial number
	Year of manufacture
	Type BF: The device, cuff and tubing have been designed to protect the patient against electric shock.
	Bluetooth communication active.
	MReading in progress. Flashes as soon as a heartbeat was detected.

Symbols on the blood pressure monitor

Symbol	Function/meaning
	Cardiac rhythm disorder or movement during reading.
	Battery fully charged.
	Battery partly discharged.
	Replace battery as soon as possible.
	Flashing! No further reading possible. Replace battery.
	Protect against liquids

Symbols on the blood pressure monitor and in the app

Symbol	Function/meaning
SYS	Systolic blood pressure in mmHg
DIA	Diastolic blood pressure in mmHg
PUL	Pulse per minute
Arr	Reading with cardiac rhythm disorder or movement
CE 0124	Device complies with the European Medical Devices Directive.
	Device must not be discarded with household waste.
	Manufacturer
	Read instructions for use
	Polarity of power supply port

Symbols on the power supply unit

Symbol	Function/meaning
	Only for use in enclosed spaces.
	Protection class II device
	Thermal fuse
	Fuse
	Device complies with the European Medical Devices Directive
	Polarity of the mains adapter plug

Quick guide

- ➡ Insert batteries, ensure correct positioning (P. 63)
- ➡ Connect cuff to monitor (P. 69)
- ➡ Attach cuff (P. 70)
- ➡ Start (and stop) a measurement by pressing the start button (P. 73)

EN

Measured systolic, diastolic and pulse values are displayed after the measurement is completed (P. 75)

This quick guide does not replace the detailed information on the use and safety of the blood pressure monitor featured in the user instructions.

Please be sure therefore to read the user instructions!

Introduction

Dear Customer,

We are delighted that you have decided to purchase a boso blood pressure monitor. The boso brand is synonymous with optimal quality and precision and is also number 1 among professionals: 96% of all German general practitioners, physicians and internists work in practice with blood pressure instruments from boso (API survey conducted by GfK 01/2016). The experience gained from decades of use by professionals has also been incorporated into all the patient monitors for home measurement.

This instrument has also passed our strict quality control procedures and is your reliable partner for monitoring your blood pressure values.

Please read these user instructions

before using the instrument for the first time, as correct blood pressure readings can only be obtained if the instrument is operated correctly.

In these user instructions, „➡“ means an action by the user.

To get help concerning usage or maintenance please contact your dealer or the manufacturer (contact details can be found on the back cover of this manual).

An unexpected operating condition or an incidence which has worsened the health condition or could have worsened the health condition should be reported to the manufacturer immediately.

If the instrument is to be sold, ensure that these user instructions are enclosed.

Introduction

The boso-medicus system blood pressure monitor is suitable for patients with an upper arm circumference of between 22 and 48 cm.

The use of accessories not referred to in these instructions for use can undermine safety.

Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 12 inch should be kept from such devices.

The Bluetooth radio transmission of the device conforms to the Radio Equipment Directive (RED) 2014/53/EU.

Purpose

Non-invasive recording of systolic and diastolic blood pressure and pulse rate in adults. Not suitable for use on newborn babies and infants. The device is suitable for self-measurement at home. The patient is regarded as the operator.

The instrument can be used with any arm circumference as shown on the corresponding cuff.

Blood pressure values

To determine the blood pressure, **two values** need to be measured:

- **Systolic (upper) blood pressure:**

This is produced when the heart muscle contracts and blood is forced into the blood vessels.

- **Diastolic (lower) blood pressure:**

This is present when the heart muscle expands and fills with blood again.

Blood pressure measurement values are expressed in mmHg (mm of mercury).

The World Health Organization (WHO) has defined the following cut-off values for assessing blood pressure:

	systolic	diastolic
High blood pressure (hypertension)	over 140 mmHg	over 90 mmHg
Borderline normal blood pressure	130 to 139 mmHg	85 to 89 mmHg
Normal blood pressure	120 to 129 mmHg	80 to 84 mmHg
Optimum blood pressure	up to 119 mmHg	up to 79 mmHg

Blood pressure values

Whether medical treatment is necessary depends not only on the blood pressure, but also on the patient's risk profile. Please contact your GP if one of the values (systole, diastole) consistently exceeds the cut-off point for high blood pressure.

Self-measurement of blood pressure often produces somewhat lower values at

home than at the doctor's. The German Hypertension League has therefore defined lower cut-off values for home measurements:

Home measurement:	135/85 mmHg
Measurement in the doctor's surgery:	140/90 mmHg

Starting up

The device must be installed and put into operation in accordance with the instructions in this manual.

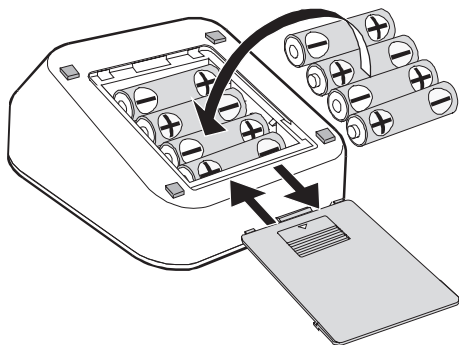
Illustration 1

- ➔ Use only high quality leak-proof batteries of the correct specifications (see "Technical data" P. 93).

⚠ Never mix old and new batteries or different makes.

⚠ If the batteries are not inserted correctly, the monitor will not work and this may cause the batteries to heat up and leak, damaging the monitor.

➔ Inserting the batteries

The battery compartment is located on the underside of the monitor. Insert batteries in the battery compartment as shown in illustration 1.

- ➔ Remove the batteries if the monitor is not to be used for any length of time.

⚠ Batteries that are short-circuited can become hot and cause burning.

Starting up

The monitor has a battery status display (see P. 55)

Battery fully charged.

Battery partially charged.

Replace battery soon.

Flashing! No further measurements possible.
Replace battery.

Care for the environment

Do not dispose of used batteries and charger units in domestic waste. You can take these to collection sites for used batteries or special waste. Contact your local authority for information.

General instructions for self-measurement

1. Fluctuations in blood pressure are quite normal. Even when measurements are repeated, there can be marked differences. Single or irregular measurements do not provide reliable information about the actual blood pressure. A reliable evaluation is only possible if regular measurements are taken under comparable conditions and the measurement values entered in the blood pressure record card.
2.  Self-testing does not mean self-treatment. Do not on **any** account change the dose of medicine prescribed by your doctor of your own accord.
3. Irregular heartbeats can affect the accuracy of the monitor or result in incorrect measurements (see page 88).
4. Incorrect measurements can also occur in patients with a weak pulse (possibly in patients wearing a pacemaker, for example). The blood pressure monitor itself has no effect on the pacemaker.
5. The device has not been validated for use on pregnant women.
6. Always take the measurement in a calm and relaxed state.
You are recommended to check your blood pressure twice a day – in the morning on getting up and in the evening once you have relaxed after work.
7. Always measure your blood pressure from the arm with the higher blood pressure values (where not required otherwise (see page 71)).

General instructions for self-measurement

To do this, measure the blood pressure first in both arms and thereafter always from the arm with the higher blood pressure.

A circular logo with the letters "EN" inside, set against a dark gray square background.

General Notices for Bluetooth Connection

-  The blood pressure monitor may not be used in locations where wireless communication is prohibited, such as in aircraft or hospitals. This blood pressure monitor can have unfavorable effects on electronic devices or electrical medical devices.
- The maximum communication distance between the blood pressure monitor and the receiver equals approx. 10 meters. This distance can be reduced by the environment; therefore, verify whether the distance is short enough for transmission of data after the measurement.
- Ensure that the space between the blood pressure monitor and the mobile device is unobstructed. The communication distance can be shortened by the construction of the building or other obstacles. Communication through reinforced concrete walls in particular may not be possible under certain circumstances.
- During pairing, Bluetooth must be deactivated on all other devices within range.
- If the improbable case should occur that this blood pressure monitor disrupts other wireless communication devices, use the blood pressure monitor at a different location.

General Notices for Bluetooth Connection

- Do not use the Bluetooth connection in the vicinity of an IEEE802.11g/b/n-WLAN device or other wireless communications systems (particularly in the range of 2.4 GHz) or near devices which emit electromagnetic waves (such as microwave appliances). This can cause the connection to be frequently interrupted, communication speed to be significantly reduced and errors to occur.

Preparing to measure your blood pressure


Note on cuff sizes:

⚠ The monitor may only be used with the following cuff types. These should be chosen according to the upper arm circumference printed on them.

Type	Arm circumference	Order number
CA01	22 – 32 cm	143-4-764
CA02	32 – 48 cm	143-4-757
CA04	22 – 42 cm	143-4-765*

* Standard cuff supplied as standard.

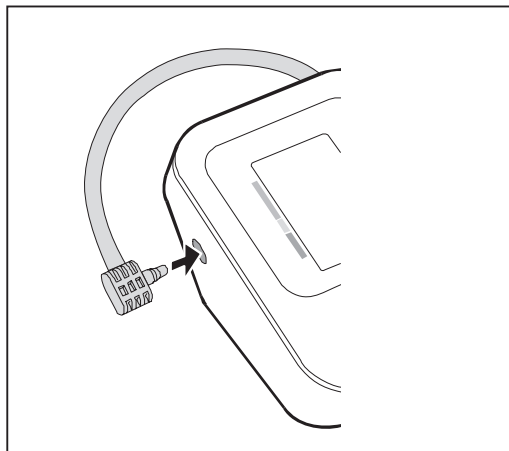Attach the cuff to the monitor by pressing the plug (click) into the socket on the left side of the unit. (See illustration 2).

EN

Illustration 2

Preparing to measure your blood pressure

Observe the following instructions when measuring your blood pressure:

Avoid smoking or drinking coffee for one hour before measuring your blood pressure.

EN

Sit comfortably for the blood pressure measurement. Support your back and arms. Do not cross your legs. Place your feet flat on the floor.

Relax for 5 minutes before taking a measurement.

Do not move during the measurement.

Attaching the cuff

The measurement should be taken from the bare upper arm.

In the case of tightly fitting outer clothing, ensure that the blood supply to the arm is not cut off when the sleeve is rolled up (if necessary remove the article of clothing before taking the measurement).

➔ Open out the cuff into the shape of a ring and slide it over the upper arm until the bottom edge of the cuff is about 2-3 cm above the elbow. The cuff must be positioned so that the marking is over the artery (see illustration 3).

Preparing to measure your blood pressure

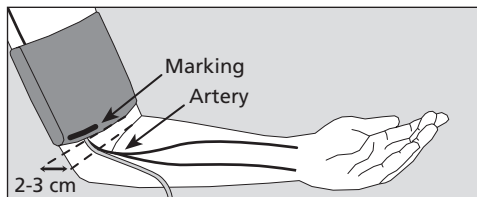
Illustration 3

⚠ Do not attach the cuff over wounds as this can cause further injuries.

⚠ Ensure that the cuff is not attached to an arm where arteries or veins are undergoing or have undergone medical treatment (e.g. shunt).

⚠ For women who have had a breast amputated, do not attach the cuff to the arm on the amputated side of the body.

⚠ During the measurement, medical apparatus that are being used on the same arm at the same time, may malfunction.

⚠ The cuff must not be too tight. There should be enough space to fit roughly two fingers between arm and cuff.

⚠ The metal ring should **never** lie over the artery (see illustration 3) as this may produce a false reading.

➡ Grasp the end of the cuff that feeds though the metal ring and wrap it around the outside of your arm. Attach the Velcro fastener by applying gentle pressure.

EN

Preparing to measure your blood pressure

- ➔ Place your arm with the cuff attached on the table, holding it relaxed and slightly bent so that the cuff is level with your heart.

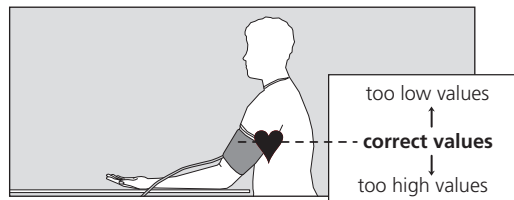
Illustration 4

Do not talk during the measurement.

- ⚠ Check that the air tube is not kinked during the measurement. This could cause congestion of the blood flow and subsequent injury.

- ⚠ The blood pressure measurement should not prevent the flow of blood for an unnecessary length of time (> 2 minutes). If the monitor fails to function correctly, remove the cuff from the arm.

- ⚠ Excessive frequent measurements can cause injury by restricting blood flow.

- ⚠ The device may not be used by unsupervised children.

- ⚠ Do not use the device near infants. This can lead to accidents or damage.

- ⚠ Do not start the device without putting on the cuff.

- ⚠ There are small parts that may cause a choking hazard if swallowed by mistake by infants.

⚠ If the ambient temperature changes significantly, leave the device about an hour to adapt to the new ambient temperature before using it.

⚠ The performance of the device can be affected by excessive temperature, humidity or altitude.

Measuring your blood pressure

➔ Start the measurement with the start button

⚠ Now hold the arm absolutely still and do not talk.

All the items on the LCD display appear briefly to indicate that the monitor is now ready for use (see illustration 5).

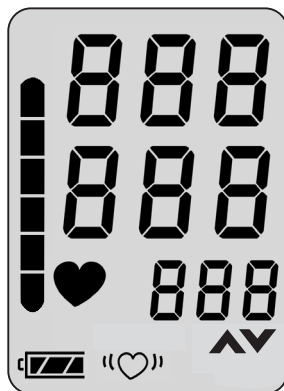
Illustration 5

Measuring your blood pressure

The pump starts to inflate the cuff.

The monitor has an intelligent automatic inflation system for gentle inflation to the correct cuff pressure.

The rising cuff pressure is displayed as a digital numerical value and in the form of a bar on the left-hand edge of the display.

When the **required pressure** is reached, the pump switches off and the air is released slowly from the cuff.

Interrupting a measurement:

A measurement can be interrupted at any stage by pressing the start button; the cuff deflates automatically.

During the measurement phase the descending cuff pressure and the flashing ♥ symbol are displayed (see illustration 6).

Illustration 6

Measurement display

After the end of the measurement, the in-built valve opens automatically and the cuff deflates rapidly.

The measured blood pressure values (systole, diastole, pulse) are displayed (see illustration 7).

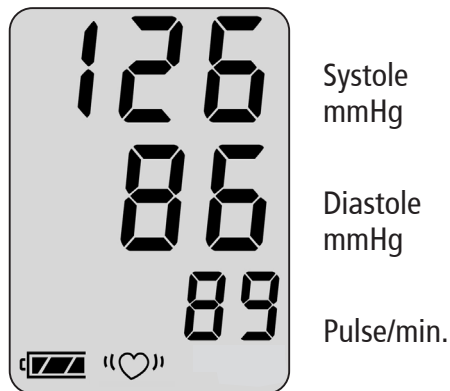
Illustration 7

If the symbol "♡" appears after the measurement, it is recommended that the measurement is repeated, holding the arm completely still. If the "♡" symbol appears again even with the arm held still, this may be an indication of an irregular heartbeat. Please discuss this with your doctor at your next visit.

EN

Blood pressure is a dynamic parameter and can be affected by the patient's position, e.g. sitting, standing, lying, moving, before or during the measurement, physical condition (stress, disease, etc.).

➔ If the measurement result is obviously wrong, repeat the measurement.

Measurement display

The unit switches itself off automatically after about 1 min. To repeat a measurement, press the start button again **briefly**.

Allow at least two minutes between measurements.

➡ If no further measurement is required, remove the cuff from the arm.

Set up app and create connection

Requires interaction with boso-App

Requires interaction with boso medicus system

- Download the boso app from the Google/Apple Store.

To enable data exchange between the blood pressure monitor and your mobile device, follow these steps:

- Ensure that Bluetooth is activated on your mobile device. (Bluetooth 4.0 is required)

When installing the app, the operating system (Android/iOS) will ask for permission to access the current location. You must answer „YES“ in order to avoid BT connection errors.

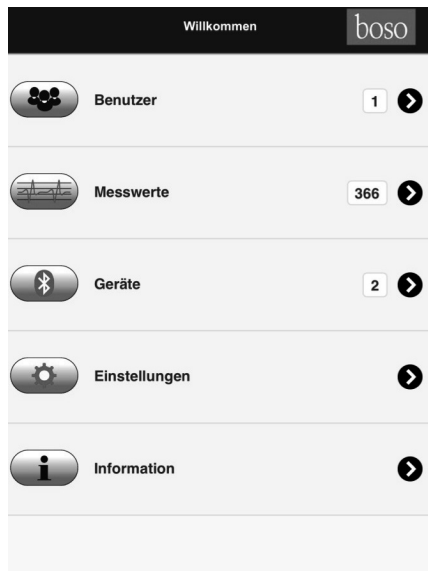
Note: The information „current location“ is not required by the boso app but by the operating system for the Bluetooth connection.

Set up app and create connection

- Open the boso app on your mobile device.
- Set up a new user:
 1. Select "User" from the main menu.
 2. Select "+New" from the bottom menu bar.
 3. Enter your personal data. Compulsory fields are indicated by " * ".
 4. Click on the "Save" button in the bottom menu bar to save the data you have entered.
- Tick the user for whom you wish to save the measurement values.

EN

Set up app and create connection

- Use the “Back” button in the top menu bar to return to the main menu.
- Open “Devices” in the main menu.

- Press the START key on the blood pressure monitor and hold it down until “Pr” appears on the display. The communication indicator is shown on the display for approximately one minute.

- In the “Devices” menu, select “Add new device” in the bottom menu bar.

If you are using iOS, you must also press “Pair” to confirm the pairing request.

- Date and time are automatically synchronised by the mobile device.

- After pairing successfully, „End” appears in the display on the monitor and the monitor is displayed in the device list in the app.

Setup is now complete. Now leave the menu via the “Back” button in the top menu bar.

Set up app and create connection

The blood pressure monitor must be reconnected to the mobile device **after changing the batteries**, as the time information is deleted when the batteries are changed.

If this is not done, the current system time of the mobile device is stored as the time stamp for the readings during data transfer.

Please proceed as follows:

- Delete the blood pressure monitor from the devices list in the app
- Delete the blood pressure monitor from the Bluetooth settings of the mobile device
- Pair again as described.

Measuring and transferring blood pressure

Now put on the cuff and carry out a blood pressure measurement as described in the user instructions.

As long as Bluetooth is active and the app is running, the measured values are automatically transferred after the blood pressure measurement has been completed.

After successful transfer, the measurement value is deleted from the measurement value memory of the blood pressure monitor.

Repeat the pairing process if the measurement data have not been transferred. If the app or Bluetooth are inactive, up to 30 measurement

values are automatically saved in the blood pressure monitor (If the measurement value memory of 30 measurements is full, the oldest measurement is deleted and the current measurement is saved. Faulty measurements are not saved) and sent after the next measurement once the app has been activated.

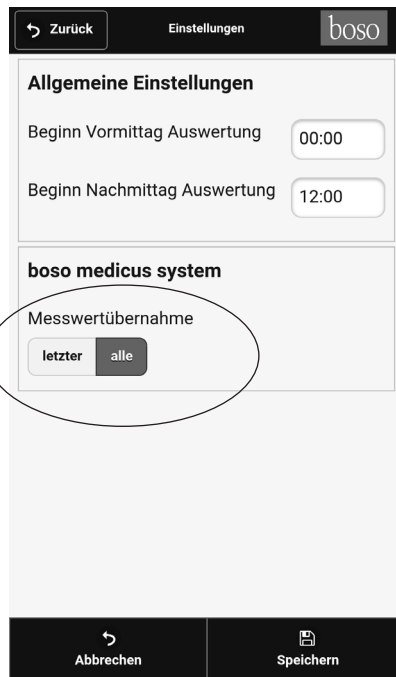
In order to enable all saved measurement values in the medicus system measurement value memory to be transferred to the app, measurement value transfer of all measurement values must be activated in the „Settings“ menu (default setting). If „last“ is selected

Measuring and transferring blood pressure

here, only the last measured measurement value is transferred and all saved measurement values are deleted.

Times for the morning and afternoon evaluations can also be altered in this menu.

Save your settings by using the "Save" button in the bottom menu bar.

The screenshot displays the 'Einstellungen' (Settings) screen of the 'boso medicus system'. At the top, there is a navigation bar with a 'Zurück' (Back) button and the title 'Einstellungen'. Below this, the 'Allgemeine Einstellungen' (General Settings) section is visible, containing two time selection fields: 'Beginn Vormittag Auswertung' (Morning evaluation start) set to '00:00' and 'Beginn Nachmittag Auswertung' (Afternoon evaluation start) set to '12:00'. The 'boso medicus system' section is highlighted with a circle, showing the 'Messwertübernahme' (Measurement Transfer) option. Under this option, there are two buttons: 'letzter' (last) and 'alle' (all). The 'alle' button is currently selected. At the bottom of the screen, there is a dark navigation bar with two buttons: 'Abbrechen' (Cancel) and 'Speichern' (Save).

Evaluating and sending measurements

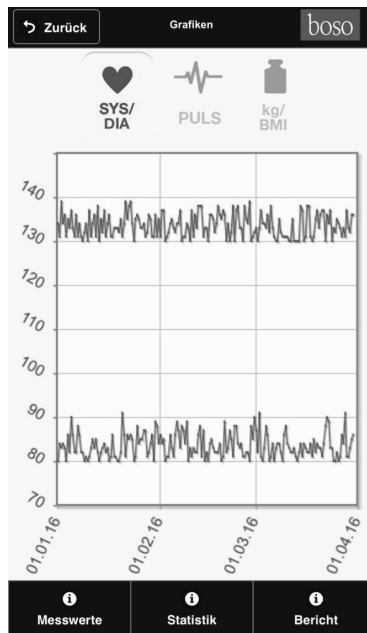
All measurement values that have been sent so far are displayed in the "Measurement values" menu.

Use the "Graphs" button in the bottom menu bar to see a graphic representation of the measurement values.

The buttons in the top area of the screen allow you to select the graphs available: "SYS/DIA" and "PULSE". Click on a graph to enlarge or shrink it.

"kg/BMI" is intended for scales.

Evaluating and sending measurements

Use the "Statistics" button in the bottom menu bar to see a statistical summary of the measurement values. The time range for the summary (date from/to) can be selected.

EN

The "Report" button in the bottom menu bar takes you to the menu which you use to send measurement values. The summary can be sent by email in PDF or XML format. The XML file can be imported into the PC program „boso profilemanager XD Home.“ This program offers additional evaluation options and is available free of charge from the Internet (<http://www.boso.de/download/profilmanagerXDHome/>).

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An:
Q

Kopie:
Q

Email
Text:

Als XML senden Als PDF senden

Deleting measurement values

Individually selected measurement values or measurement data older than 1/3/6/12 months can be deleted via the "Delete" button in the bottom menu bar.

Note that data cannot be restored. Please archive your data in good time by creating an XML or PDF file using the "Report" button.

Zurück		Messwerte Max Mustermann					boso
	SYS	DIA	PUL	Arr	kg		
17.10.2018 11:09:00	-	-	-	-	85	☐	
30.03.2016 18:00:00	136	86	66	-	-	☐	
30.03.2016 08:00:00	-	-	-	-	78,5	☐	
30.03.2016 08:00:00	136	85	68	-	-	☐	
29.03.2016 18:00:00	-	-	-	-	78,8	☐	
29.03.2016 18:00:00	132	83	67	-	-	☐	
29.03.2016 08:00:00	-	-	-	-	78,7	☐	
29.03.2016 08:00:00	133	81	65	-	-	☐	
28.03.2016	-	-	-	-	78,9	☐	
Statistik		Grafiken		Bericht		Eingabe Gewicht	Löschen

EN

Mains operation

The mains connection socket is located at the rear of the monitor.

Where required, use only the bosco power supply unit (**order number 410-7-150**). This power supply unit produces a rectified output of the correct polarity. Other commercially available power supply units can cause damage to the electronic components, which will invalidate the warranty. There is also a risk of fire with these other power supply units.

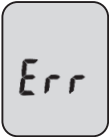
 Do not touch the batteries, the power supply unit socket and the patient at the same time.

 Remove the power supply unit plug from the blood pressure monitor when it is not in use.

Error messages

If any problems occur during the measurement, the display will show an error message instead of a blood pressure reading.

Meaning of error messages:

Cause of Err message:		Troubleshooting:
	Excessive variation in cuff pressure during measurement	Hold arm still.
	No valid pulse readings	Check position of cuff (see page 70) and repeat measurement.
	Systole – diastole difference too small: Systole – diastole ≤ 10 mmHg	Check position of cuff (see page 70) and repeat measurement.
	Incorrectly inflated	Cuff possibly too loose, attach cuff more firmly (see page 70)

Error messages

EN

Cause of Err message:		Troubleshooting:
	The pulse value could not be determined correctly.	Check position of cuff and repeat measurement.
	Internal Error	Remove the batteries and briefly press START button. Put the batteries back into the monitor. If the error persists, return the instrument for repairs to the manufacturer.
		
	"♥" Display:	Troubleshooting: Repeat measurement recommended, holding the arm absolutely still. If the "♥" symbol appears again, even with the arm held still, this may be an indication of an irregular heartbeat. Please discuss this with your doctor at your next visit.

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Cause of **Err** message: Troubleshooting:

Bluetooth connection error

Remove the batteries and briefly press **START** button. Put the batteries back into the monitor. Carry out the pairing again. If the error persists, return the instrument for repairs to the manufacturer.

When installing the app, the system will ask for permission to access the current location. You must answer „YES“ in order to avoid BT connection errors.

EN

Cleaning and Disinfection

Cleaning Monitor:

Use a soft, dry cloth to clean your monitor.

Cleaning Cuff:

Small stains on the cuff can be removed carefully with a proprietary washing-up liquid.

Disinfection Cuff:

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend the disinfectant antiseptic liquid (Schülke & Mayr). To disinfect the cuff, we recommend spray disinfection. In particular, if the device is used on multiple users it is important to pay attention to regular cleaning and disinfections of the cuff.

Warranty Conditions

We give 3 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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boso accepts no responsibility for losses, such as operational disruptions or data losses, which can occur due to the use of this blood pressure monitor.

No guarantee is made that this blood pressure monitor can establish a connection with all Bluetooth-compatible devices.

Customer service

Customer service:

Warranty and repair work must be undertaken by trained and authorised personnel. Do not modify this equipment without authorization of the manufacturer.

Send the monitor, carefully packaged and with sufficient postage, to your authorised dealer or directly to:

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Disposal

Do not dispose of monitors and batteries in the domestic waste.

At the end of its lifespan, the monitor must be taken to a collection site for obsolete electronic items.

Please note that batteries and rechargeable batteries must be disposed of separately (local authority collection site).

Expected operational lifetime of the equipment: 5 years (using the unit 6 times a day).
Expected operational lifetime of the cuff: 2 years (using the unit 6 times a day).

Technical data

Measurement principle:	oscillometric
Measurement range:	Systolic: 60 to 279 mmHg Diastolic: 40 to 200 mmHg Pulse: 40 to 180 per minute
Cuff pressure:	0 to 299 mmHg
Memory store:	30 measurements
Display:	LCD
Operating conditions:	environmental temperature +10°C to +40°C Relative humidity 15 to 85% Air pressure 800 hPa to 1060 hPa
Transport/ Storage conditions:	environmental temperature -20°C to +60°C Relative humidity 10 to 95% Air pressure 700 hPa to 1060 hPa
Power supply:	DC 6 V (4 x 1.5 V Mignon IEC LR 6 alkaline manganese batteries) Alternative special option: DC 6 V power supply unit, Order No. 410-7-150

Technical data

Typical battery life:	500 measurement cycles (depending on inflation pressure and frequency of use).
Battery check:	symbol display in LCD window
Weight:	250 g without batteries
 Dimensions (WxHxD):	96 mm x 68 mm x 130 mm
Classification:	Medical device with internal energy source (in battery operation) / class II (in power supply unit operation), continuous operation mode
Applied part:	Type BF (☞)
Protection against solid objects:	IP20 IP classification is the degree of protection provided by enclosures in accordance with IEC 60529. This device is protected against solid foreign objects of 12mm diameter and greater such as fingers. This device is not protected against water.

Technical data

Clinical test:	Accuracy complies with the requirements of ISO 81060 Part 2
Bluetooth:	Bluetooth Version 4.0
Maximum deviation of cuff pressure measurement:	± 3 mmHg or 2% of the reading (whichever is greater)
Maximum deviation of pulse rate display:	± 5 %
Applicable standards:	DIN EN ISO 80601-2-30 „Medical electrical equipment - Part 2-30: Particular requirements for the basic safety and essential performance of automated type non-invasive sphygmomanometers”.
Wireless Communication:	VZ (MURATA Manufacturing Co. Ltd.) Bluetooth Ver.4.OLE BLP Frequency band: 2402-2480 MHz Maximum RF output power: 1.6dBm Modulation: GFSK

Calibration checks - Testing instructions

(for trained specialist staff only)

Calibration checks – every 2 years at the latest – must be performed by the following institutions or persons:

- Manufacturer
- Trade measurements authorities
- Persons who fulfil the requirements of the legislation governing the operation of medical devices.

A) Function testing

Function testing of the monitor can only be carried out on a person or with a suitable simulator.

B) Testing of pressure circuit integrity and deviation of pressure display

Note:

- a) When the pressure in measurement mode increases above 299 mmHg, the quick release valve is triggered and the pressure circuit opens. When the pressure in test mode increases above 320 mmHg, the measured value display flashes.
- b) For the calibration checks a special test plug must be used. The test plug is available for a fee from the manufacturer.
A pump ball must also be introduced into the pressure circuit.

Calibration checks - Testing instructions

(for trained specialist staff only)

Testing

- 1.) Remove batteries.
- 2.) Hold start button pressed down and insert batteries.
- 3.) Release start button.
The monitor is now ready for testing; the current pressure is displayed in the SYS and DIA fields.
- 4.) Perform test for deviation of pressure display and pressure circuit integrity (observe setting time for cuff – at least 30 seconds) in the usual manner.

5.) Switch off monitor by pressing the START button again.

The device must not undergo service or maintenance while it is in use.

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The screenshot shows the 'boso' mobile application interface. At the top, there is a navigation bar with a 'Zurück' button, the word 'Bericht', and the 'boso' logo. Below this, there are three main sections: 'An:' with a search input field, 'Kopie:' with another search input field, and 'Email Text:' with a large text area. At the bottom, there are two buttons: 'Als XML senden' and 'Als PDF senden', each with an envelope icon.

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(<http://www.boso.de/download/profilmanagerXDHome/>).

The screenshot shows the 'boso' application interface. At the top, there is a dark header bar with a 'Zurück' button, the word 'Bericht', and the 'boso' logo. Below the header, there are three sections: 'An:' with a search input field, 'Kopie:' with a search input field, and 'Email Text:' with a large text area. At the bottom, there are two buttons: 'Als XML senden' and 'Als PDF senden', each with an envelope icon.

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BOSCH + SOHN GmbH u. Co. KG