bosotherm primus

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Ⓔ User Instructions

Ⓕ Mode d'emploi

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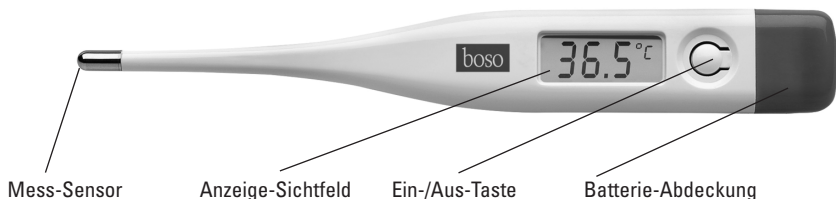
(DE) Symbole auf dem Thermometer / (EN) Symbols on the thermometer / (FR) Symboles sur le thermomètre / (ES) Símbolos en el termómetro / (IT) Simboli sullo termometro

Symbol / Symbol / Symbole / Símbolo / Simbolo	Funktion – Bedeutung / Function/meaning Fonction/signification / Función/significado Funzione/Significato
	Ein-/Aus-Taste ON/OFF-button Interrupteur marche/arrêt Botón ON/OFF Copercio
	Keine weitere Messung möglich. Batterien ersetzen. No further reading possible. Replace battery. Une mesure n'est plus possible. Remplacer la batterie. No son posibles más mediciones. Sustituir las baterías. Impossibile continuare la misurazione. Sostituire le batterie.
	Gerät vom Typ BF Type BF equipment Appareil de type BF Aparato del tipo BF Apparecchio di tipo BF
	Lotnummer Lot Number Numéro de Lot Número de Lot Numero di Lot
	Hersteller Manufacturer Fabricant Fabricante Produttore
	Gebrauchsanweisung lesen Read instructions for use Lire le mode d'emploi Leer las instrucciones de uso Leggere le istruzioni per l'uso
	Gerät darf nicht über den Hausmüll entsorgt werden. Device must not be discarded with household waste. L'appareil ne doit pas être jeté à la poubelle. El aparato no se debe eliminar en la basura doméstica. L'apparecchio non deve essere smaltito con i rifiuti urbani generici.
L °C	Temperatur unter 32 °C Temperature under 32 °C Température inférieure à 32 °C Temperatura inferior a 32 °C Temperatura inferiore a 32 °C
H °C	Temperatur über 42 °C Temperature over 42 °C Température supérieure à 42 °C Temperatura superior a 42 °C Temperatura superiore a 42 °C
CE 0124	Gerät ist konform mit der europäischen Medizinprodukterichtlinie. Device complies with the European Medical Devices Directive. L'appareil est conforme à la directive européenne « Dispositifs médicaux ». El aparato cumple la Directiva sobre Productos Sanitarios europea. L'apparecchio è conforme alla direttiva europea sui dispositivi medici.

bosotherm primus

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Messarten

Oral

60 ± 10 Sekunden
Normal
35,7 °C – 37,2 °C

Rektal

60 ± 10 Sekunden
Normal
36,2 °C – 37,7 °C

Axillar

100 ± 20 Sekunden
Normal
35,2 °C – 36,7 °C

MESSARTEN

• In der Mundhöhle (oral)

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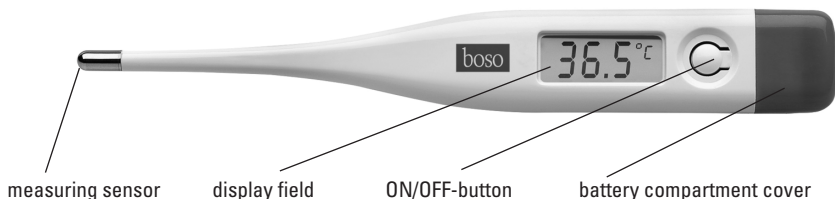
MEASURING METHODS

• In the mouth (oral):

Put the thermometer underneath your tongue. The instrument must have good contact with the tongue. Close the mouth and breath calmly through your nose in order not to influence the measurement by the inhaled air. Minimum measuring time: 60 seconds

• In the anus (rectal):

This is the most reliable method and the most appropriate for babies and children. Carefully immerse the tip of the thermometer 2-3 cm into the anus. Minimum measuring time: 60 seconds

• Underneath the arm (axillary):

Under medical aspects this measuring method will lead to inaccurate results. Therefore it is not appropriate to get exact measurements. Minimum measuring time: 100 seconds

IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is appropriate for the measurement of the body temperature only!
- This device can be used at home and in hospital.
- The user should be at least 11 years old.
- The patient can be the user.
- Do not interrupt the measurement before the signalling tone [beep] indicates that the minimum measurement time is completed.
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from tremor and shock!
- NEVER boil the thermometer!
- Only water or the disinfectants listed in the section «cleaning and disinfecting» are to be used for cleaning. The thermometer must be intact when it is immersed in liquid disinfectant.
- The thermometer has a long-life battery. For disposal, please comply with local regulations for special waste.
- Regular checks on the accuracy of the measurement system should be carried out in accordance with the valid directives of the individual countries, governing this issue.
- Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this thermometer. Therefore, a minimum distance of 11 feet should be kept from such devices.

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CE 0124

TURNING ON THE THERMOMETER

The device must be installed and put into operation in accordance with the instructions in this manual.

To get help concerning usage or maintenance please contact your dealer or the manufacturer.

An unexpected operating condition or an incidence which has worsened the health condition or could have worsened the health condition should be reported to the manufacturer immediately.

To turn on the thermometer, push the ON/OFF-button; a short signalling tone [beep] indicates «thermometer ON.» A so-called LCD-test automatically takes place to check the display. Then a «L» and a flashing «°C» appear on the upper right of the display, if the ambient temperature is less than 32°C. Now the thermometer is ready for use.

USING THE THERMOMETER

During a measurement, the temperature is displayed continuously and the «°C» symbol flashes. If the signalling tone sounds (beep-beep-beep-beep 10 times) and the «°C» no longer flashes the measurement is completed (the measured increase in temperature is now less than 0,1 °C in 15 seconds).

To extend the lifetime of the battery, turn off the thermometer after usage by pressing the ON/OFF-button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

MEMORY OF MEASURED VALUES

When turning the thermometer on, the temperature of the last measurement will be displayed. After 3 seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

CLEANING AND DISINFECTING

For cleaning of the instrument please use isopropyl alcohol (70%). For disinfecting wipe the instrument completely with Antifect Liquid (please obey 5 min. work-in-time).

△ The device is NOT waterproof!

CHANGE OF BATTERY

As soon as the symbol " □ " appears on the display, the battery is weary and a new battery has to be inserted. Open the cover of the battery compartment and insert the new battery with "+"-symbol facing upwards.

The lithium battery can get hot, burn or explode.

TECHNICAL SPECIFICATIONS

Maximum thermometer

Measuring range: 32.0°C to 42.0°C

Temp.< 32.0°C:display «L» for low (too low)

Temp.> 42.0°C:display «H» for high (too high)

Accuracy: ± 0.1°C between 35,5°C and 42°C

Operating conditions: environmental temperature 5°C to 35°C, air pressure 700 – 1060 hPa, relative humidity 40 to 88%

Liquid crystal display (LCD) with three digits. Smallest unit of display: 0.1°C

Signalling tone sounds when thermometer is ready for use and when measurement is completed.

Memory: automatic memory of the last measured value.

Storage conditions: environmental temperature

-25°C to +55°C, relative humidity 15% - 95%

Battery: 1,5 V, IEC-Type LR/SR 41

1,000 measurement cycles (depending on frequency of use).

Norm: EN 12470-3, clinical thermometer

Protection against solid objects and water: IP22 (in accordance with IEC 60529)

Expected operational lifetime of the equipment: 3 years

WARRANTY CONDITIONS

We give 2 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear, transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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Warranty and repair work must be undertaken by trained and authorised personnel. Do not modify this equipment without authorization of the manufacturer.

CARE FOR THE ENVIRONMENT

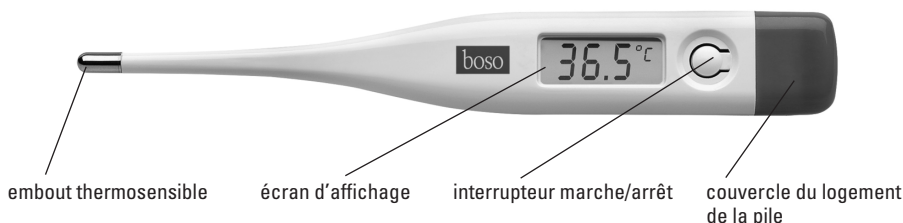
Used batteries and defective electronic devices do not belong into the household waste. 

They can be deposited at special centres. Please contact your local authorities for information.

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