**BOSCH
+SOHN**

boso

bosolux auris

Deutsch	2
English.....	7
Français.....	12
Italiano	17
Español.....	22
Polski	27
Suomi	32
EMC Guidelines.....	38

- DEUTSCH -

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- ENGLISH -

Before using the bosolux auris and its accessories please read these instructions carefully and store them safely for future reference.

bosolux auris

Intended purpose:

bosolux auris otoscopes are designed for the examination of the exterior auditory canal. In addition, the instrument can be used for other illumination purposes in non-invasive examinations.

1. Otoscope 1 x
2. Disposable ear specula, grey:
2.5 mm 10 x
4.0 mm 10 x
3. Instructions for Use 1 x

Optional accessories:

1. alkaline batteries (AAA/LR03) 2 x

Warning:

The examination may only be performed with the ear specula attached. NEVER use without the ear specula.

The ear specula must not remain in the ear (skin contact) for more than 1 minute. The pressure build-up during the pneumatic test must be carried out very carefully (risk of overpressure and resulting damage).

Use the otoscope only with boso spare parts and accessories.

The otoscope must not be shone into the eyes.

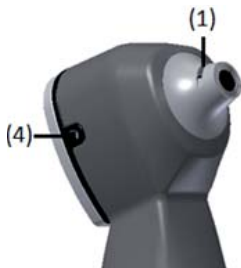
Make sure that the slot for attaching the ear specula (1) is sharp-edged and not worn.

Application:

Attaching the ear specula:

The bosolux auris otoscope may only be introduced to the ear canal with the ear specula attached.

The ear specula is attached to the otoscope tip by bayonet mount, by lightly twisting clockwise to engage with the slit (1).

The magnifying lens in the view field achieves up to 3x magnification. The viewing lens can be turned clockwise to the limit for instrumentation.

Using the pneumatic test:

The bosolux auris otoscope has a connection option for a rubber bulb with tube for a pneumatic test (4).

Cleaning:

Clean with a soft and lint-free cloth, moistened with a little alcohol if necessary. Do not dip the otoscope in liquid.

Ear specula are single use only. Re-use of single use items may cause damage to the ear specula.

Sterilization:

Sterilization is not approved.

Disposal:

The product must be disposed of at a separate collection of electrical and electronic equipment.

Storage/transport conditions and operating conditions:

Ambient temperature between -10° and +45°C.

Rel. humidity between 30% and 90%.

Power supply:

- 2 alkaline cells (AAA/LR03)

Switching on:

Press the push button (2) once.

Adjust luminosity:

Increase luminosity: While power is on press and hold the push button (2) until the desired luminosity is reached.

Decrease luminosity: Press and hold the push button (2) again until the desired luminosity is reached.

Powering off:

If the instrument is in operation press push button (2) once.

The otoscope also has an automatic switch-off function after 3 minutes.

Changing the batteries:

Unscrew the battery cover (3) and remove the batteries.

Insert new batteries (AAA/LR03) with the positive '+' pole to the instrument head. Then screw on the battery cover again. Do not change the battery when near the patient.

Technical specifications:

Nominal voltage	2.5V
Nominal current	23mA
Application part	BF
Protection class	Class II
IP code	IP20
Ambient conditions:	5° C -40° C
	10-75% air humidity
	700-1070 hPa atmospheric pressure
Power supply	2×AAA Alkaline batteries

Warnings

1. When cleaning / disinfecting devices that have come into contact with patients, make sure that appropriate detergents / disinfectants are used.
2. Make sure appropriate personal protective equipment and clothing is used, and follow all safety regulations / warnings.
3. Always remove the batteries before cleaning / disinfection. Make sure that no liquid penetrates into the housing.
4. The use of accessories not listed in the Instructions for Use may compromise safety.
5. During use, metal components near the instrument head may heat up. This occurs in particular if the device is switched on for an extended period of time. Do not leave the unit on when it is not in use.
6. Do not attempt to service the otoscope while using it on the patient.
7. Pay attention to all significant risks of reciprocal disturbance which may arise when using the otoscope during special examinations or treatments.
8. To avoid or minimize electromagnetic interference between the otoscope and other devices, please observe the instructions of the other devices.
9. It is not unusual for the product to change from standby to operating mode due to electrostatic interference. As long manual switch-off or automatic switch-off occurs after 3 minutes, it can return to standby mode.
10. If the patient experiences a sensation of heat at the contact points during the examination, the diagnosis must be temporarily stopped and the otoscope switched off for cooling.
11. Contraindications:
 - Avoid direct contact with mucous membranes or injured skin.
 - Do not shine into the eyes.

Date of purchase

Explanation:

	Type BF Applied Part
	Class II Equipment
	Operating Instruction
	Stand-By
	Do not reuse
	Manufacturer
	Date Of Manufacture
	Caution
	Electric Scrap
	In Conformity with EU Guidelines

Warranty Terms

This product is covered by a 2 year factory warranty from date of purchase. The purchase date is to be verified by presenting the invoice. Within the warranty period, defects resulting from material or manufacturing defects shall be remedied free of charge. The warranty does not extend the warranty period for the entire device, but only for the replaced components.

The warranty does not include wear and tear, transport damage and any damage that arises due to improper handling (e.g. failure to observe the Instructions for Use) or due to handling by unauthorized persons. No claims for damages against us are justified by the guarantee.

The purchaser's statutory claims due to defects under Section 437 of the German Civil Code BGB are not restricted.

In the event of a warranty claim, the device must be sent, together with the original proof of purchase, to:

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Customer service:

Maintenance and repair work may only be carried out by trained and authorized personnel. No alterations may be made to the device without the manufacturer's permission. Post the device carefully packed and with sufficient postage to your authorized dealer or directly to:

BOSCH + SOHN GmbH u. Co. KG
Service department
Bahnhof St. 64
72417 Jungingen,
GERMANY
T: +49 (0) 7477 / 92 75-0

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EMC Guidelines

Below cables information are provided for EMC reference.

Cable	Max. cable length, Shielded/unshielded		Number	Cable classification
Charger USB cable to bosolux auris (only for bosolux auris)	1m	unshielded	1 Set	DC Power

Important information regarding Electro Magnetic Compatibility (EMC)

bosolux auris needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual and other documents; bosolux auris conforms to this EN 60601-1-2:2007+AC:2010/IEC 60601-1-2:2007 standard for both immunity and emissions.

Nevertheless, special precautions need to be observed:

- The bosolux auris is only to be used with a medical approved charger, plug and USB cable;
- The use of accessories and cable other than those specified, with the exception of accessories and cables sold by bosolux of bosolux auris as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY or decreased LIFESPAN of the bosolux auris.
- bosolux auris should not be used adjacent to or stacked with other equipment. In case adjacent or stacked use is necessary, The bosolux auris should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacturer' declaration - electromagnetic emissions - for *bosolux auris*

Table 1:Guidance and manufacturer's declaration – electromagnetic emissions

The bosolux auris is intended for use in the electromagnetic environment specified below. The customer or the user of the bosolux auris should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The bosolux auris uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The bosolux auris is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Compliance	

Table 2:Guidance and manufacturer's declaration – electromagnetic immunity

The bosolux auris is intended for use in the electromagnetic environment specified below. The customer or the user of the bosolux auris should assure that it is used in such an environment.

Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge(ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC 61000-4-5	±1 kV lines to lines ±2 kV lines to earth	±1 kV lines to lines ±2 kV lines to earth	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) For 0,5 cycle 40% UT (60% dip in UT) For 5 cycle 70% UT (30% dip in UT) For 25 cycle <5% UT (>95% dip in UT) For 5 cycle	<5% UT (>95% dip in UT) For 0,5 cycle 40% UT (60% dip in UT) For 5 cycle 70% UT (30% dip in UT) For 25 cycle <5% UT (>95% dip in UT) For 5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of bosolux auris requires continued operation during power mains interruptions, it is recommended that the bosolux auris be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

Note: U_i is the a.c. mains voltage prior to application of the test level.

Table 3: Guidance and manufacturer's declaration – electromagnetic immunity

The bosolux auris is intended for use in the electromagnetic environment specified below. The customer or the user of the bosolux auris should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80MHz</p> <p>3 Vrms 80MHz to 2.5GHz</p>	<p>3V</p> <p>3V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the SYSTEM, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80\text{MHz to } 800\text{MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800\text{MHz to } 2.5\text{GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts(W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Notes

Notes

www.boso.de