
boso Stethoskop

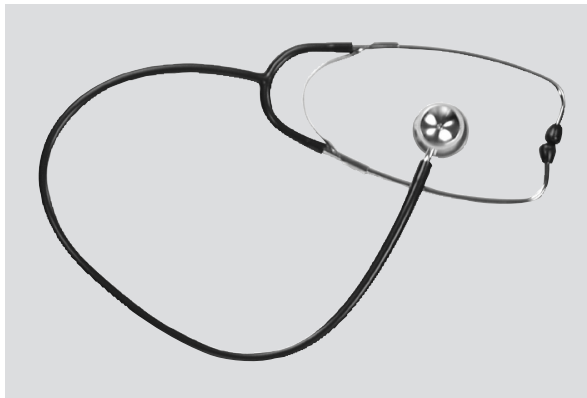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DE

2

**User
Instructions**

EN

8

**Mode
d'emploi**

FR

14

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IT

20

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ES

26

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DE

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EN

General information

- Your boso-stethoscope conforms to the requirements of the European Directive 93/42/EWG (CE-mark).
-  Please read this instruction manual carefully before using the stethoscope and store it afterwards.
- Do not expose the stethoscope to extreme heat or cold.

- Do not steam sterilize the stethoscope.
- Do not use the stethoscope without diaphragm.
- Do not put the stethoscope into cleaning liquid.
- Only use spare parts of bosco.
- The stethoscope can be damaged by sharp or pointed instruments.
- Do not stick the eartips too deep into the ear otherwise you may hurt yourself.

10 General use

EN

- The stethoscope is a diagnostic instrument for auscultation.
- The stethoscope should be used by competent persons only.
- After placing the eartips into your ears, the chest piece has do be put on the relevant part of the body with careful pressure.

Using the chest piece (only Boli, Boki)

- The stethoscope can be adjusted from bell to the diaphragm or vice versa as the chest piece can be turned 180° both directions. A clicking sound can be heard as soon as the chest piece is engaged correctly on either the bell or the diaphragm.
- Depending on the body part which has to be auscultated either the diaphragm or the bell has to be used. The diaphragm is used for the auscultation of high frequencies, the bell is used for low frequencies.
- Place the eartips into your ears and gently sweep the diaphragm with the finger tip to test whether it is already active.

12 Changing the eartips

EN

- Soft eartips as well as hard eartips can be used on the binaural to adapt to the optimal acoustic sensitivity and comfort of the stethoscope. To change the eartips simply screw the required type onto the binaural.
- To avoid injuries, please check the correct position of the eartips on the binaural.

Cleaning

For cleaning use a soft cloth with mild soapy water or with alcohol (70% Isopropyl). The diaphragm must be cleaned only with a soft cloth with alcohol (70% Isopropyl).

Never use abrasive cleaning solution, thinner or benzene for cleaning.

Disinfection

For disinfectant wipes (at least 5 minutes exposure time) of the device, we recommend the disinfectant liquid Microzid Liquid Sensitiv (Schülke & Mayr).

We give 3 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear, transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Da-

mages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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BOSCH + SOHN GmbH u. Co. KG